




## Fitting Protocol

Basic Sling  
Models 25045, 25054, 25070

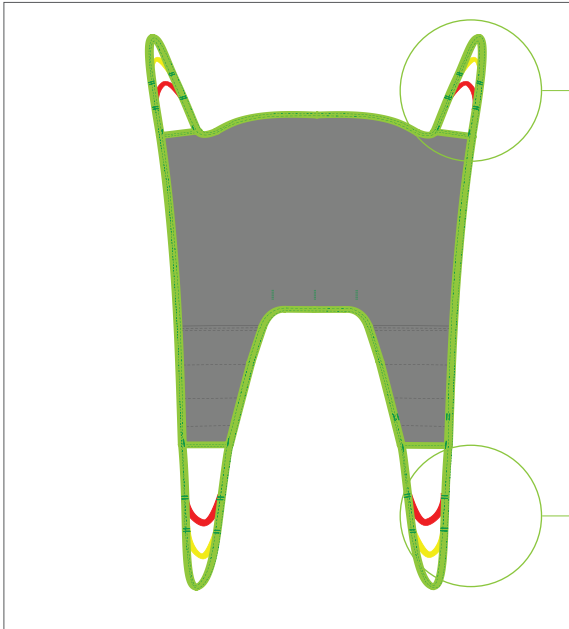

- Green Strap
- Yellow Strap
- Red Strap

**Extension Loops**

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.



- Green Strap**  
Increased Recline  
Less hip flex and greatest angle
- Yellow Strap**  
Semi-Recumbant  
Mid-hip angle
- Red Strap**  
Upright Position  
Least hip angle

- Green Strap
- Yellow Strap
- Red Strap

**Extension Loops**

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Match colours on both sides of the patient for a 90 degree angle in the hips.

**Maximum User Weight: 300kg/660lb**


- Model 25045 - Polyester Net**
- Model 25054 - Soft Air Mesh**

- Model 25070 - Polyester**


**Transfer Situation:**



**Leg Placement:**

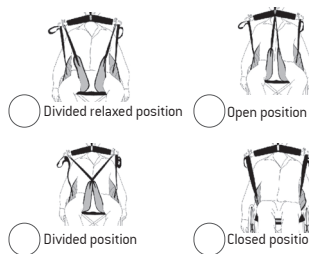

**Suspension:**

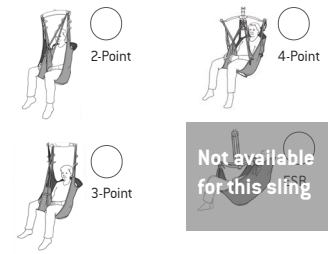

**Available sizes:**

Indicate size required

- JS
- JM
- S
- M
- L
- XL

**Accessories required:**



- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large



- 25102 Reinforced Leg Support - One Size



- 50866 Upholstered Cover for Hanger Bar  
**Do not use in wet environments.**

**Patient Information:**

Name	
Facility	Room
Lift Used	Hanger Bar Used

**HUMAN CARE SWEDEN (HQ)**

Årstaängsvägen 21B  
117 43 Stockholm  
Phone:- +46 8 665 35 00  
Fax:- +46 8 665 35 10  
info.se@humancaregroup.com

**HUMAN CARE CANADA**

10-155 Colonnade Road  
Ottawa, ON K2E 7K1  
Phone: +613.723.6734  
Fax: +613.723.1058  
info.ca@humancaregroup.com

**HUMAN CARE NEDERLAND**

Elspeterweg 124  
8076 PA Vierhouten  
Phone: +31 577 412 171  
Fax: +31 577 412 170  
info.nl@humancaregroup.com

**HUMAN CARE UNITED STATES**

8006 Cameron Road, Suite K  
Austin, TX 78754  
Phone: +512.476.7199  
Fax: +512.476.7190  
info.us@humancaregroup.com

The Basic Sling is a comfortable sling without reinforcements and handles. The sling adapts to the patient and offers good support for the entire back. The Basic Sling can be used in most lifting situations, fits most patients and is a good choice if it is not established who is going to be using the sling. The Basic Sling is offered in Polyester Net and Soft Air Mesh fabric that allow moisture and heat to transfer efficiently from the patient. This allows the sling to be left behind the patient for a short period of time after the transfer is completed. The Polyester material is durable, easy to apply and maintain.

### Additional Instructions

To staff/caregiver:


To patient:


Fitting Protocol Completed by:

Name		
Date	Position/Title	



**Important!** This fitting protocol does not replace the user manual. Read all instructions before use!