HUMAN CARE

Your Life. Your Way.

Rollator neXus 3

User manual/Bruksanvisning/ Manuel de l'utilisateur/ Gebrauchsanweisung 470018, 470022, 470024, 479022, 479024

Innehåll

SVENSKA

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User Manual

Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

INFORMATION symbol indicates recommendations and information for proper, trouble-free usage.

WEIGHT CAPACITY symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

CE MARKING symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

MEDICAL DEVICE symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

RECYCLE according to local regulations.

READ INSTRUCTIONS and ensure you fully understand them before using this product.

Product modification is not permitted.
Before use, always check that the sling is not damaged or worn.

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at www.humancaregroup.com.

Contact your local distributor if you have any questions about the product and its use.

Notice to user/patient in case of serious incident

Any serious incident that has occurred in relation to the product, should be reported to the local contact, who reports to the manufacturer, and the competent authority of the country in which the user/patient is established.

Compliances and Standards

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO 11199-2:2021.

About our products

On delivery

Check the following before using your new product:

1. Check that no damage has occurred during transport (if this is the case, please immediately inform the delivery agent).
2. Check that the number and type (s) of products detailed on the delivery certificate agree with the goods delivered.
3. Check that the delivery is of the desired model and type of product.

Please contact us immediately in case of any delivery problem.

Return or refund

If any product has to be returned for any reason, please contact us to obtain a claim number and advice on how to proceed.

The goods should be returned in complete and undamaged condition in their original packaging.

The following information should accompany any product on its return. The product's identification number, delivery date, the date the user took possession of the aid, the date of the product's last service, the person whom we should contact about the fault, whether it be the user or another, and the nature of the fault.

Human Care HC AB reserves the right to inspect the returned product. If no fault can be found, the item will be returned to the user. Transport costs may then be chargeable.


Product lifetime

The product's expected lifetime is seven (7) years, if the product is used as intended and maintained according to the manufacturer's instructions, depending on the intensity of use and maximum load applied during use. If the product label is no longer legible, the product should be discarded.

Product label

The product label contains information about the rollator.

The label is located on the lower vertical frame tube, above the right rear wheel.

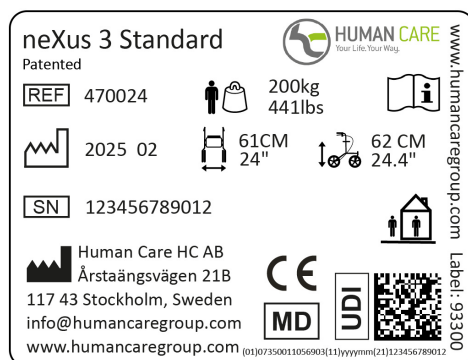	Article number
	Maximum user weight
	Date of manufacture
	Address of manufacture
	Unique Device Identification, including (01) EAN code, (11) production date, (21) serial number
	Serial number

	Read the manual
	Product width
	Indoor / outdoor use
	CE marking
	MD Medical Devices

Additional Warnings

Ensure that all bolts are securely fastened before using the rollator!

DO NOT adjust brake tension until wheels lose molding sheen!


Brakes must be in the parked/locked position before sitting on the rollator!


DO NOT hang anything on backstrap or it may break!

Rollator neXus 3

Warnings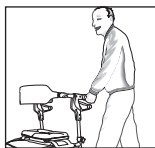

DO NOT leave your rollator unattended. Anytime the rollator has been out of view (eg. placed in vehicle, etc.) inspect the rollator completely for any signs of damage before use.

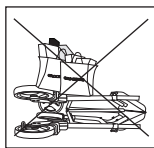DO NOT put anything on your folded rollator as this will cause damage to the frame, plastic parts, etc.

DO NOT use as a wheelchair/transport chair. The rollator has not been designed for this purpose.


If the rollator has been stored in any type of transportation, such as a plane, train, boat, bus, car, etc. you risk damage to the rollator. Please inspect very carefully before usage.


Be sure the brakes are locked before sitting on seat. Caution! Pinch point: **Do not put fingers between brake and push handle!**

DO NOT lean on one side of the rollator when sitting down. Distribute weight evenly to prevent the rollator from tipping over.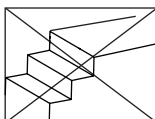


DO NOT use on stairs or escalators. Only use this product on even surfaces clear of obstructions.

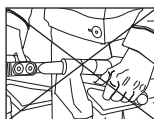DO NOT use brake handles when sitting and standing. Use upper handgrip only to stabilize your body.

DO NOT squeeze brake continuously while walking as this will cause premature wear to the brake pad and tire.

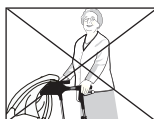DO NOT rest anything on back-strap as damage may occur.

Touch the back of your legs to seat before sitting.

DO NOT exceed weight capacity!


Care and Use

DO NOT use chlorine bleach or abrasive cleansers. To clean, wipe down with a damp cloth using a mild detergent only.

INSPECT your rollator every year by an authorized dealer/service agent. Only let authorized personnel adjust your rollator.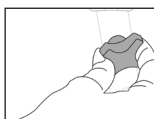

CHECK periodically to ensure that the black height adjustment knobs are tightened securely.

CHECK your rollator frequently for any loose, cracked or corroded plastic or metal components.


CHECK your rollator to ensure the front forks and all wheels swivel easily and are clear of any dirt or obstructions.

DO NOT store the rollator near a heat source.

DO NOT leave your rollator exposed to the elements (weather).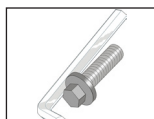

Use only authorized parts/accessories with your walking aid.

This is a walking aid only designed to assist you with mobility. Only use the seat for rest periods.

NEVER use the rollator for any other reason other than for its intended purpose.

Cleaning and service

When cleaning, water and mild detergent are recommended.

When washing the rollator, remove the two soft seat cushions by loosening the red Nexus handle and unbuttoning the cushions from the seat. After washing, the cushions are mounted on the seat again. Machine wash 60 ° C for 10 minutes can be used, drying max 70°C for 10 min, observe the finishing requirements. Avoid flushing with bearings (wheels, forks).

NOTE!

Do not use naphtha, gasoline or other solvents.

Rollator neXus 3


Intended Use

The neXus 3, in all its models, are stable walking aids, suitable for use both in- and outdoors. It has a cable free brake system, which makes it easy to perform maintenance on. The brake system decreases the risk of jamming brake wires. The Nexus 3 has a unique way of folding which allows it to be stored upright whilst folded and it can be folded and unfolded by simply using the handle. Nexus 3 has a wide and soft seat that makes it comfortable to sit on.

Article numbers:

Product Name	Part no.	Maximum user Weight
neXus 3 Super Low		
Black	470018 *	200 kg / 441 lb
Blue	470018B	
Red (Burgundy)	470018R	
neXus 3 Low		
Black	470022	200 kg / 441 lb
Blue	470022B	
Red (Burgundy)	470022R	
neXus 3 Standard		
Black	470024	200 kg / 441 lb
Blue	470024B	
Red (Burgundy)	470024R	
neXus 3 Low Wide		
Black	479022	250kg / 551lbs
neXus 3 Standard Wide		
Black	479024	250kg / 551lbs

TO SIT, touch the back of your legs to the edge of the seat to ensure position of rollator before sitting down.

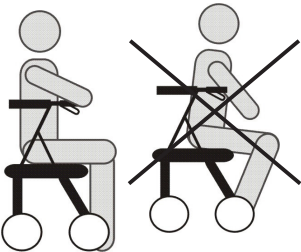
IMPORTANT! Test to be sure rollator will not roll before attempting to sit. Brakes must be in parked/locked position.


This rollator is designed for use as a walking aid and should never be used for any other purpose.

DO NOT support your weight on the handles when attempting to sit. Use the handles to stabilize your body as you carefully turn around. Keep your hands on the rubber handles and slowly lower your weight onto the seat.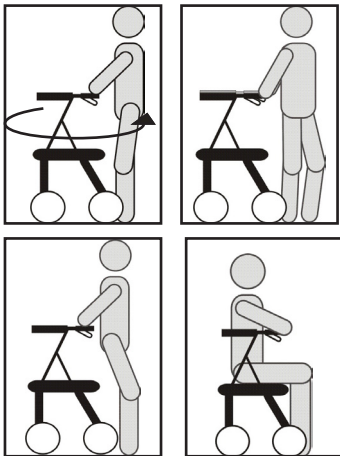

Ensure when walking your path is clear and free of obstacles to avoid injury and damage.


Assembly

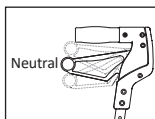Ensure handle is in neutral position.


Slide upper tube into lower tube. **Important:** Upper internal tube colour must match colour of lower tube in frame!

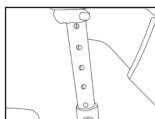
Set handle to desired height. **Important:** Brake handle must be in neutral position!

Align hexagon shape of bolt in hexagon shape in plastic cap. **Important:** The holes in the upper and lower internal tubes will not line up perfectly!

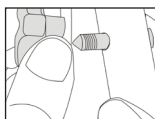
Using the pointed end of the bolt, place it in the hole in the push handle. At the same time, tap the brake pad and slowly push through the bolt.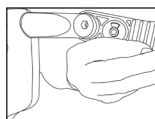


To install backstrap, ensure key shape faces up. Place into clamp located at end of handle. Tighten into place with screw and nut.


Installing the soft zippered bag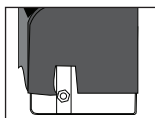

Slide the metal frame into the slots on each side of the soft bag. Each frame side will only fit on the appropriate soft bag slot. The clamp should be visible in the soft bag cut out.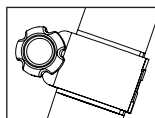

Attach each clamp to the front rollator frame as shown. Make sure that each side of the bag is level and that the clamp is on the side above the fork cap.

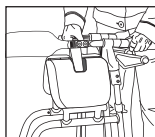Note: To simplify the assembly of the soft bag, start by slightly folding the rollator by pulling up on the seat handle.

Unfold the rollator once clamps are secured.

Take care not to strip the clamp screws when tightening!


Maximum bag capacity: 11lb/5kg

Rollator neXus 3

Accessories


Basket - Model 4510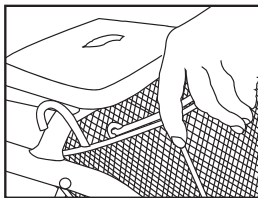


Add the basket to the crossbar supporting the seat using the hooks.

*Basket does not fit neXus 3 Wide models.

Cane Holder - Model 80703

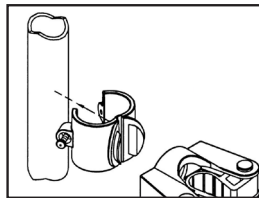
Open the clamp enough so that it may be secured around the frame tubing. As you open you will see the clamp locks go towards each other.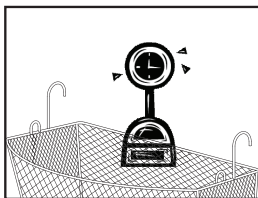


Caution: DO NOT overload basket.

Maximum Load: 10lb/5kg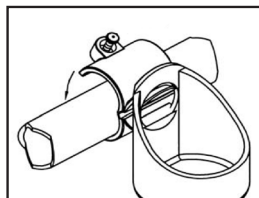


Before closing the clamp, take the attachment/cup and place into the clamp locks. Once the clamp is secured, these locks will hold the attachment/cup securely in position.

Tray - Model #80702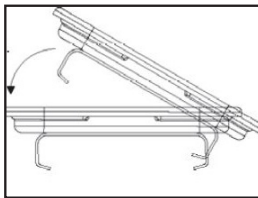


Attach the bracket of the tray over the back edge of the seat. Secure in place by firmly pressing it over the front edge of the seat. Remove by lifting.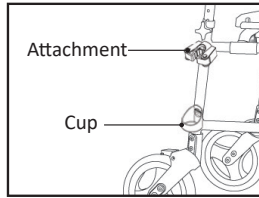

Note correct position of attachment and cup.

Optional Accessories


Cup Holder - Model #90394

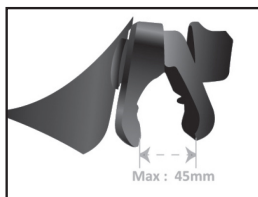
Gently unscrew the rotating knob clockwise to loosen. Note: Maximum 45mm

Test to ensure stability before use.

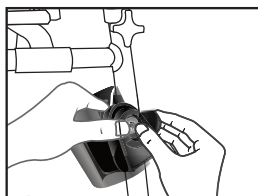
Place the clip on the rollator frame under the star handle with the cup facing towards the user.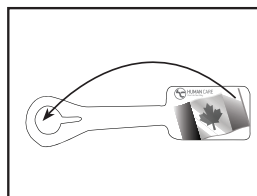


Identification Tag - Model #90392/90393

Wrap the identification tag around the area to which you will affix the tag. Bend the tag slightly and insert the wide edge of the identification tag through the keyhole opening.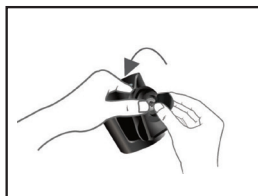

Tighten the rotating knob counter-clockwise.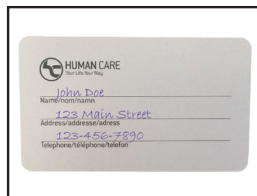


Enter information on card provided with tag and insert into sleeve.

Rollator neXus 3

Assembly instructions Slow Down Brake, art. nr.: 93122

These assembly instructions explain how to mount the Slow Down Brake on the neXus rollators rear wheels. The Slow Down Brake is delivered in pairs, a brake for the right rear wheel and a brake for the left rear wheel.

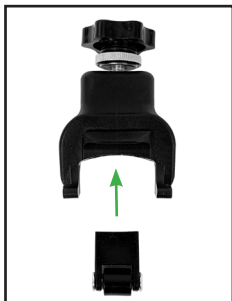1. Ensure the small wheel cassette is inserted in the slowdown brake during the assembly.


2. Take the slow down brake and loosen it by unscrewing the knob, so the small wheel cassette goes deeper into the slowdown brake.


3. Lay the rollator down on its back frame, with the front wheels facing upward into the air. The rear wheels should be fixed on the floor and parallel to each other.


4. Place the slow down brake diagonally against the rear wheel.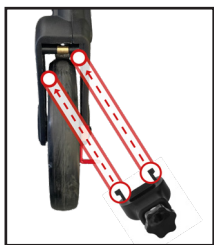

5. The clips on the slowdown brake should be inserted diagonal into the fork frame.

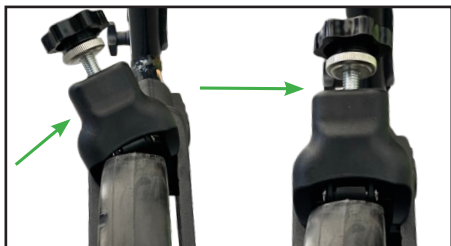
6. When the clips are inside the frame, twist the slow down brake into the fork frame, it should click into place.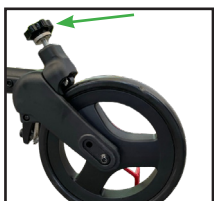

7. Once the slow down brake is in place, push the knob forward and listen for another *click*.

8. Repeat the process for the second wheel.

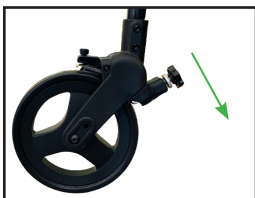
9. Stand the rollator up and push the slowdown brakes towards the floor to ensure that they are securely attached.

Disassembly instructions


1. First disassemble the back fork by removing the four screws shown in picture.
2. Remove the slow down brake from the fork.
3. Re-assemble the fork by mounting the four screws again.


Rollator neXus 3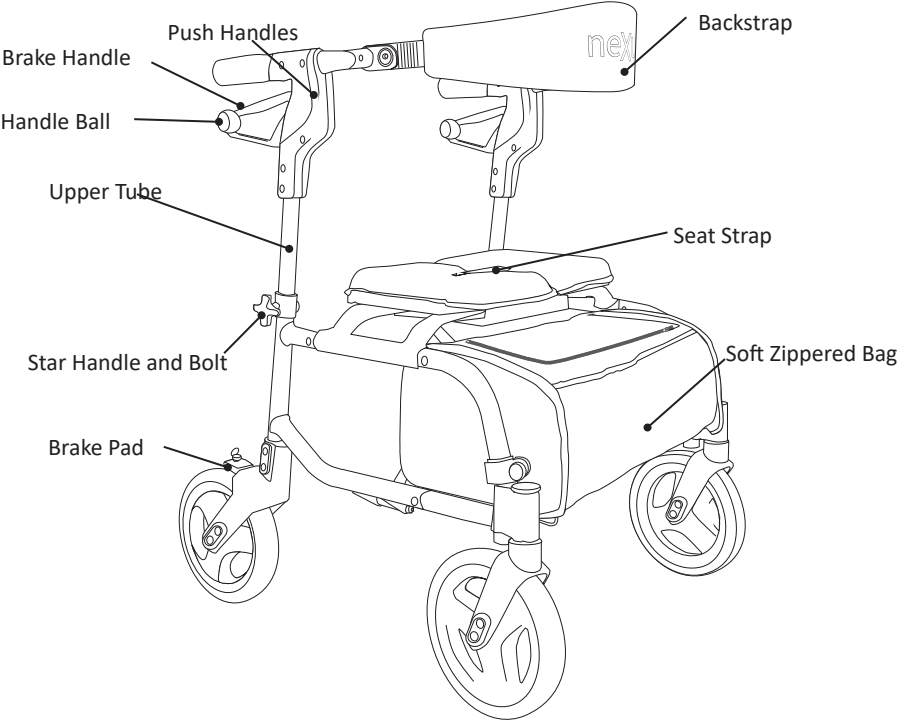


Handle Height Adjustment

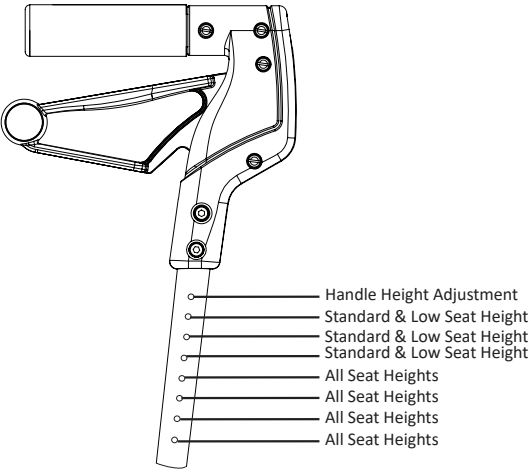Brake and Usage/Adjustment

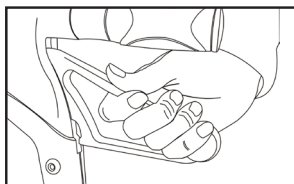
LOCK THE BRAKE by pushing down on the handle (for the loop handle, push down on the red ball located at the end of the handle). The handle will lock into a downward position when properly engaged.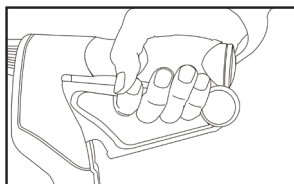

RELEASE THE BRAKE by pulling up on the handle.

BRAKE FOR SPEED CONTROL by squeezing on the handle.

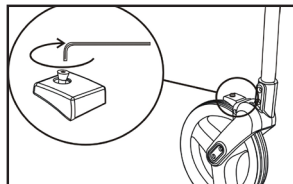ADJUST THE BRAKE by turning the screw on the brake pad using an Allen Key.

CLOCKWISE to tighten brake.

COUNTER CLOCKWISE to loosen the brake.

IMPORTANT ADJUSTMENT WARNINGS

Damage may occur if:


- Handle not in neutral position during adjustment!
- Hexagon shape in bolt does not line up with hexagon shape in plastic cap!

- Handle is twisted back and forth when inserting bolt!

NOTE: DO NOT OVERTIGHTEN!


Warning!
Clamp risk!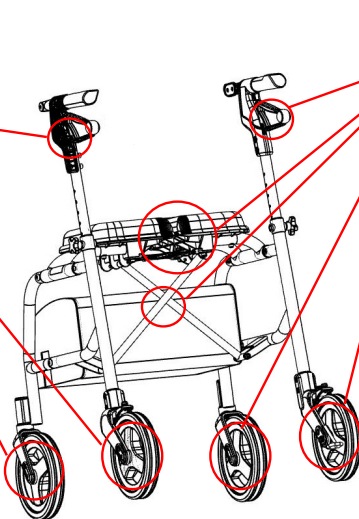


Warning!
Clamp risk!

Rollator neXus 3

Opening/Folding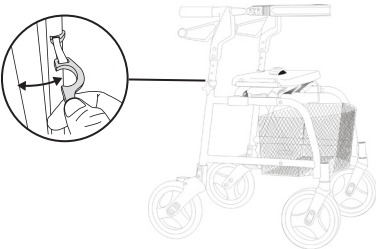

Loosen the larger plastic part on the lock snap and slide the rollator's handle outwards.

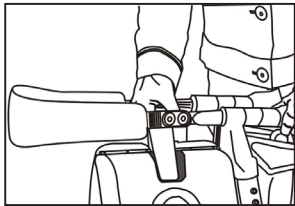↑ TO FOLD
PULL UP on the seat strap located in the middle of the seat.
↓ PUSH DOWN on the seat to lock into place.

← PULL APART push handles to open the rollator into the proper walking position. Raise the rollator to rest on the rear wheels while pulling apart.

CAUTION!

 Never sit on the seat unless it is locked into place and the brakes are in the parked/locked position.

IMPORTANT!

The nexus can be partly folded for passing the rollator through narrow doorways.

Always use the seat center handle to fold the rollator. Folding the rollator in any other manner may cause injury.

If storing rollator for extended periods of time in folded position, raise handles to avoid indentation on seat.

Serial Number
(Located on Right Rear Leg) _____
Date of Purchase _____

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