**HUMAN CARE**  
Your Life. Your Way.

# Rebel

Reconditioning Manual/Rekonditioneringsanvisning/  
Manuel de maintenance

# Content

---

**Reconditioning Manual (ENG) 4**

Maintenance, inspection and test 4

Warnings and recommendations 7

Exploded view 7

Spare Parts 8

**Rekonditioneringsanvisning (SV) 11**

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Sprängskiss 14

Reservdelar 15

**Manual de maintenance (FR) 16**

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Vue éclaté 19

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## Maintenance, inspection and testing

ENG

To ensure that your walking aid is functioning properly and safely, an annual inspection should be done only by an authorized agent. The following maintenance should be done regularly:

### ○ **Cleaning**

In accordance with local regulations where required, cleaning may be done by industrial washing at 60 degrees for 10 minutes. Pressure washing must only be done in a controlled environment by professional cleaners. Do not wash the bearings with a pressure washer. Regular cleaning with a mild liquid detergent or damp cloth is recommended. DO NOT use any toxic cleaning materials on the product as it will damage it.

**Wash only using an industrial washer/sterilizer (see image)**

***Note! Do NOT use thinner, petrol or other solvents.***

### ○ **Post treatment**

Should cleaning be performed in an industrial setting, ensure all connections, spaces and holes are dried completely before usage. Lubricate any moving parts. For home cleaning (wiping with a damp cloth), ensure the rollator is free of residue moisture to avoid rusting its components.

### ○ **Finish**

The rollator is painted and sealed for a superior finish, however scratches can occur with impact. Scratches in the frame may result in corrosion.

### ○ **Corrosive components**

The rollator does contain components that may rust if left in a moisture setting. Periodically check all welding points and screws. It is recommended that an authorized agent perform this inspection. If corrosion is present, replace parts. If corrosion is extensive, replace rollator.

### ○ **Periodic inspections**

At minimum, it is recommended that an authorized agent perform an annual inspection of the rollator and its components to ensure it is functioning as intended. Do not use the rollator if any components are damaged or inoperable. Have an authorized agent perform necessary repair and/or replacement.

### ○ **Handle grip**

Periodic replacement of the handle grip may be necessary with usage. Replace as required.

### ○ **Braking system**

The brakes should function in both the control and park position and the wires should be intact. Should these control functions not operate as intended, cease usage and immediately have the rollator repaired.

- Height adjustment**  
Control function.
- CE-marking**  
Ensure that the CE-marking is present and legible.
- Reflectors**  
Ensure that the reflectors are still there and visible.
- Folding mechanism**  
Control function
- Wheels**  
Ensure the wheels are not worn out and spin easily to ensure proper movement of the rollator. If any noise is heard, have an authorized agent inspect product.
- Front forks**  
Make sure that the ball bearings are firmly attached and that the forks swing easily. Test by swinging both wheels at a minimum of three times and up to ten times. There should be no more than two swings difference between the two wheels while they swing between 3 and 9 o'clock. Adjust the inertia by replacing and adjusting the fork screws M12x56, .
- Seat**  
Periodically check to ensure that the seat mechanism and screws are firmly in place. If any components are not secure, do not sit on the rollator and have it inspected for repair immediately. If used, ensure the seat pad is properly mounted in the seat place for maximum comfort.
- Stability**  
The wheels should all rest firmly on the ground and that there should be no distortion of the frame.
- Walking Test**  
Before using your rollator, take a small walk with it to ensure it is properly functioning.

**Authorized Dealer/Service Agent**

**Last Service Date**

## **Intended use**

This rollator was designed to act as a support device while walking or as a resting seat while stationary and locked into position. It should never be used as a transport device. Never attempt to alter the rollator in any way.

## **Note!**

**Reconditioning must be done only by an authorized agent. If incorrectly performed by an unauthorized individual, it could result in an accident/injury.**

To locate an authorized agent, contact your local office:

Europe: [info.se@humancaregroup.com](mailto:info.se@humancaregroup.com)

Canada: [info.ca@humancaregroup.com](mailto:info.ca@humancaregroup.com)

United States: [info.us@humancaregroup.com](mailto:info.us@humancaregroup.com)

Australia: [info.au@humancaregroup.com](mailto:info.au@humancaregroup.com)

Netherlands: [info.nl@humancaregroup.com](mailto:info.nl@humancaregroup.com)

**When exchanging parts on the rollator, use only Human Care authorized parts to ensure continued CE-validation.**

**If corrosion is more than a film of rust, discard and recycle rollator according to local regulations.**

**Never use this rollator in any way other than for its intended purpose.**

**It is strongly recommended that a therapist or doctor assess your needs prior to using this product.**

**Inspect your rollator frequently prior to usage to ensure there is no damage or wear.**

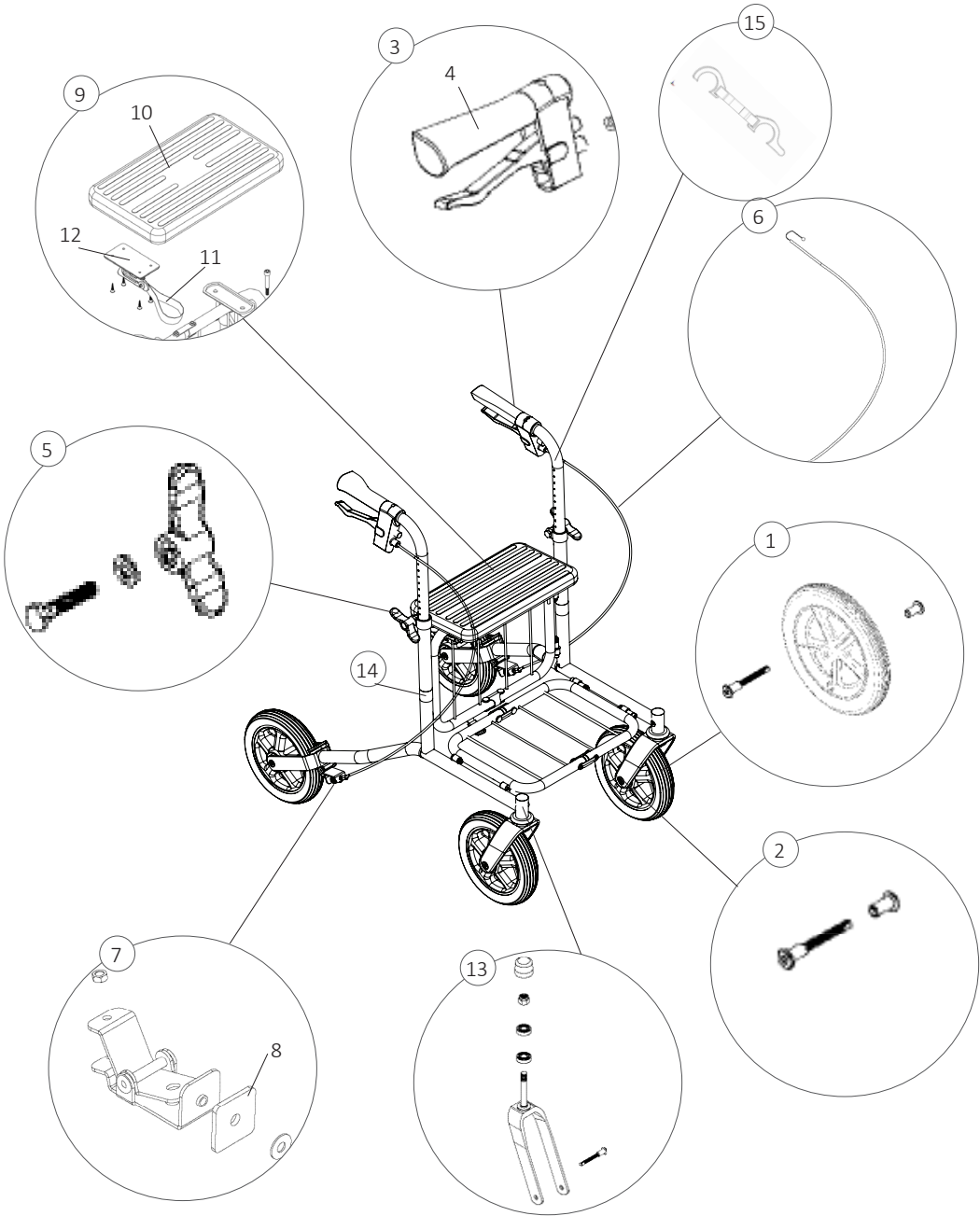
**User must not exceed this product's weight capacity!**

Rebel

140kg/309lb

Rebel Reinforced

250kg/551lb

No	Designation (Qty/Rollator)	UofM	Art. no	Additional Instructions
1	<b>Wheel kit Rebel (4 required)</b> Complete wheel Wheel nut (2/kit) Axle bolt	EA	89015	Axle bolt and nut must always be replaced when changing wheels.
1	<b>Wheel Rebel Reinforced (4 required)</b> Wheel Wheel nut (2/kit) Axle bolt	EA	89086	
2	<b>Axle bolt and nut Rebel (4 required)</b> Wheel nut Axle bolt	EA	89032	
2	<b>Axle bolt and nut Rebel Reinforced (4 req'd)</b> Wheel nut Axle bolt	EA	89091	
3	<b>Handle kit Rebel (2 required)</b> Handle tube Complete brake Ergonomic handle	EA	89055	Left or right side. For Rebel High 57 and 62 use 92903 For Rebel High 67 and 72 use 92953
3	<b>Handle kit Rebel Reinforced (2 required)</b> Handle tube Complete brake Ergonomic handle	EA	89033	Left or right side. For Rebel High 57 and 62 use 92904 For Rebel High 67 and 72 use 92954
4	Ergonomic handle	EA	20-024	
5	<b>Wing nut complete (2 required)</b> Wing nut Screw Washer	EA	50937	Lubricate when needed.
6	<b>Complete wire (2 required)</b> Wire housing Wire Wire lock Wire end	EA	89050	
7	<b>Brake kit (2 required)</b> Inner brake housing Outer brake housing Pop nut Wear washer Screw (M6x6) Washer Adjustment screw Spring Screw (M6x16) Wheel nut Axle bolt	EA	89095	For Reinforced model, part 89091 must also be replaced when replacing the brake handle.
8	Wear washer	EA	10-004	Turns when worn.

9	<b>Seat kit (1 required)</b> Seat Gate lock Folding loop handle Screw (M6x20 - 2/kit) Gate lock screw	EA	89020	
10	<b>Seat</b> Seat Screw (M6x20 - 2/kit)	EA	89070	
11	Folding loop handle	EA	10-009	
	Tube protection	EA	50949	
12	<b>Gate lock kit (1 required)</b> Gate lock Screw (4/kit) Folding loop handle	EA	10-018	
13	<b>Front fork kit Rebel (2 required)</b> Front fork Bearing (2/kit) Wheel nut Axle bolt Cap Cover Screw (M12x56) Screw (M6x7) Washer Locking Washer	EA	89025	
13	<b>Front fork kit Rebel Reinforced (2 required)</b> Front fork Wheel nut Axle bolt Screw cover Screw (M12x45)	EA	89045	
14	Reflector strip	EA	00028	
	Touch-up paint	EA	F2	
15	Locking snap		73004	

## SPARE PARTS FOR OLDER MODELS

	<b>Front fork kit (2 required)</b> Front fork Wheel nut Axle bolt Screw cover Screw (M12x45)	EA	89045	For old standard and reinforced models manufactured until 2009-07.
	<b>Front fork kit (2 required)</b> Cover Locking Washer Distance Washer Spring Washer Front fork Bushing (2 required)	EA	20-014 00111 00136 00130 20-030 200-006 20-026	For Rebel lows manufactured until 2013-03. Parts must be ordered separately.
	Cover	EA	70886	From 2013-04.

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