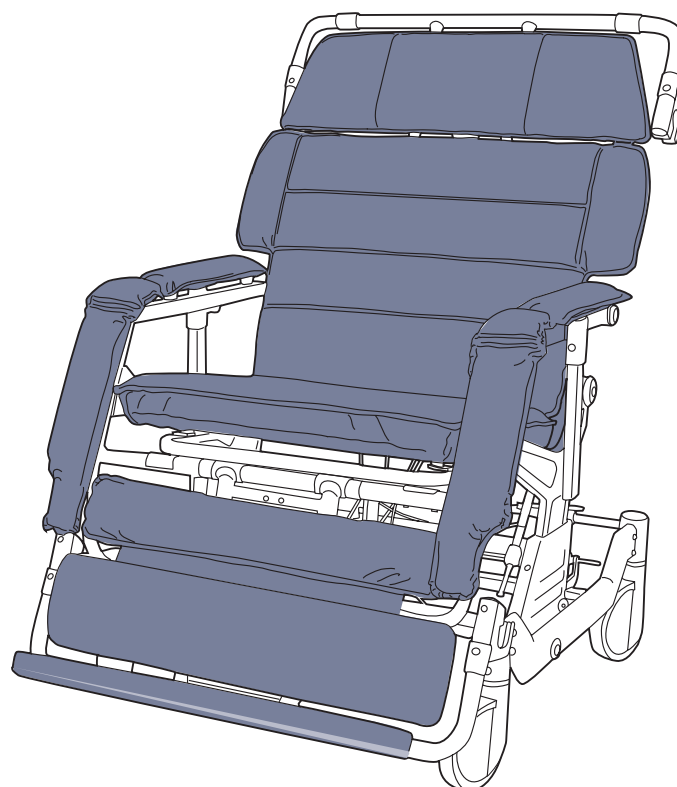
HUMAN CARE

Your Life. Your Way.

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Barton I-series, Convertible Chairs Human Care/

Barton I-series, Fauteuils gériatriques convertibles de Human Care

Contents

User manual	4
Safety Instructions	4
Symbols	4
Intended Use	4
General Care and Use	4
General Warnings	5
Warnings	5
Recommendations	5
Product Benefits	5
Product Description	5
Prior to Usage	6
Components	6
Safety Warnings	6
Product Label and Symbols	8
Usage Guidelines	8
Adjusting the Seat Height	8
Levers	8
Adjusting the Headrest	8
Adjusting the Head Support	9
Adjusting the Armrests	9
Adjusting the Footrest	10
Caster Functions	10
Adjusting the Brackets	10
Operating the Chair	11
Tilting	11
Sitting to Supine	11
Supine to Sitting	11
Trendelenburg	12
Cleaning Recommendations	12
Material	13
Troubleshooting	13

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Safety Instructions

Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

INFORMATION symbol indicates recommendations and information for proper, trouble-free usage.

WEIGHT CAPACITY symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

CHLORINE symbol indicates usage of approved chlorine on product is permitted.

Symbol indicates hand washing only.

Symbol indicates item may be wiped with approved disinfectant.

CE MARKING symbol indicates product meets requirements of EC directives and can be sold in North America.

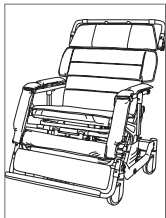
MEDICAL DEVICE symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

RECYCLE: Once the product has reached the end of its life, it should be disposed according to your country's rules, especially for the separate collection of electrical and electronic products. Correct disposal helps prevent negative consequences for the environment and human health.

Intended Use

Convertible Chair

Human Care's Convertible Chair is a mechanical positioning chair that can be adjusted to various positions. The device is used to alter postural positions and to provide a means of transferring a patient horizontally from a height adjustable bed or trolley to the chair and vice versa. The chair can then be transformed from a supine position into the sitting position without ever lifting the patient.

Its construction is made of steel tubular components, on to which is fitted foam-filled upholstery. The mechanism allows the chair to transform from the horizontal supine position into a chair, which can then be tilted. The reclining function allows the chair and patient profile to change from a sitting position to a horizontal or supine position suitable for transferring. The Tilt-in-Space function allows the I-400 / I-700 to be independently inclined to the rear (Tilt-in-Space).

In addition to the features of the standard I-series, I-400x adds real-time hydraulic height adjust and a forward tilt function to improve ease of patient transfer, patient ingress/egress, etc.

Notice to user/patient in case of serious incident

Any serious incident that has occurred in relation to the product, should be reported to the local contact, who reports to the manufacturer, and the competent authority of the country in which the user/patient is established.

Product lifetime

The product's expected lifetime is five (5) years, if the product is used as intended and maintained according to the manufacturer's instructions, depending on the intensity of use and maximum load applied during use. If the product label is no longer legible, the product should be discarded.

General Care and Use

Check the Convertible Chair prior to each use to ensure it is not damaged in any way. Do not use if there are any signs of wear and damage on part of the chair.

Follow cleaning instructions as indicated.

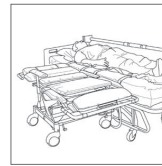
Only qualified persons should perform patient transfers. Use caution during transfer.


Do not attempt to repair the chair yourself or modify it in any way.

Do not use this equipment to transfer patients in a moving vehicle of any type.

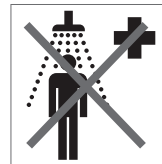
Use only the Human Care patient transfer system designed for use with the I-series positioning and transfer chairs.

This product is intended for indoor ambient environments. It is not intended for use in high moisture areas (i.e. showers, pool therapy), extreme temperature, humidity or pressure environments.

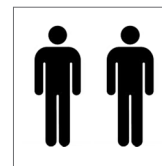Facility administrator and/or risk management department should approve all patient handling protocols associated with the Convertible chair prior to use.

Some patients' physical limitations or weights present the potential for injury. When in doubt that an attendant or patient may be injured, ensure additional attendants are present.

Prior to using on a patient, it is strongly recommended that training or practice with other caregivers is done until the user is comfortable with using the product.

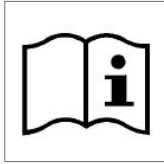
Assess the patient's ability to handle the move prior to using the Convertible Chair.

General Warnings

The decision to use the Convertible positioning and transfer chair must be made by a trained professional. In all cases, this chair can only be articulate when the patient is attended.

If you are unable to understand the warnings, cautions and/or instructions, contact a healthcare professional, dealer or technical personnel (if applicable) before attempting to use this equipment. Using this equipment without fully understanding it may result in injury or damage.

Therapists and other healthcare professionals should determine if the patient requires a seat positioning strap prior to use to ensure safe operation of this equipment by the user. Serious injury can occur in the event of a fall.

Warnings

DO NOT USE the product if,

Weight capacity is exceeded.

Product is damaged in any way.

You have not read and understood all transfer procedures.

You have not read and understood all warnings and instructions.

Recommendations

Do not use Convertible Chair without adequate prior training.

Increase safety by evaluating transfer environment prior to transfer.

There is always risk when transferring a patient. Evaluate patient ability prior to transfer.

To avoid risk of caregiver injury, have two caregivers perform the transfer.

Product Benefits

For the caregiver/facility

- Simplifies job conditions - efficient and easy to use
- Reduces patient length of stay due to early mobilization
- Injury prevention - driving force for SPH initiatives

For the patient

- Safe and effective transport
- Increased comfort as no sling application is required
- Faster recovery

Product Description

Technical Specifications

Item	Part no.	Seat Width	Overall Width (cm/inch)	Weight Capacity
I-400	B1041-TS-GEL	21"/53cm	30"/76cm	181kg/400lbs
I-400X	B1042-TS-GEL	21"/53cm	31"/79cm	181 kg/400 lbs
I-700	B1071-TS-GEL	25.5"/65cm	35"/89cm	318kg/700lbs

Item	Part no.	Overall Length (Prone Position)	Overall Length (Wheelchair Position)
I-400	B1041-TS-GEL	72.5"-78.5"/184-199cm	45.5"-62.5"/115-159cm
I-400X	B1042-TS-GEL	69.5"-82"/177-208cm	44"-56.3"/112-143cm
I-700	B1071-TS-GEL	72.5"-78.5"/184-199cm	45.5"-62.5"/115-159cm

Item	Part no.	Overall Height (Prone Position)	Overall Height (Wheelchair Position)
I-400	B1041-TS-GEL	27.5"/70cm	51"/130cm
I-400X	B1042-TS-GEL	27"-34.5"/68-88cm	52"/132cm
I-700	B1071-TS-GEL	27.5"/70cm	51"/130cm

Headrest adjustment: 0-5.9"/0-15cm (I-400X), 0-5"/0-12,7cm (I-400, I-700)
 Footrest adjustment: 0-5"/0-12,7cm (I-400, I-700), 0-5.9"/0-15cm (I-400X)
 Frame height adjustment: 3.9"/10cm (I-400)
 Foot pedal height adjustment: 7.9"/20cm (I-400X)
 Diameter castors: 5.9"/15cm (I-400, I-400X- I-700)
 Tilting angle 0-17° (I-400, I-700), 0-15° back/0-15° forwards (I-400X)
 Back angle: 0-70° (I-400, I-400X, I-700)
 Product weight: 86,6kg/191lbs (I-400), 142,4kg/314lbs (I-400X), 93,4kg/206lbs (I-700)

Standards and compliance

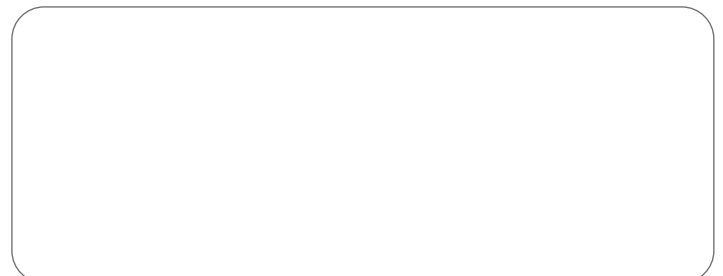
Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO7176-1:2016, EN12182:1999, CA117:2013, EN1021-1:2006, EN1021-2:2016 and ISO10993-1:2018.

Authorized Dealer

Prior to Usage

Before use verify that:

All casters work properly (4-caster lock, 4-caster free, steering lock).

The equipment reclines smoothly and stops in any desired position.

The height and angle of the footrest and the height, angle and depth of the headrest can be adjusted properly.

The side rails can be locked and released reliably.

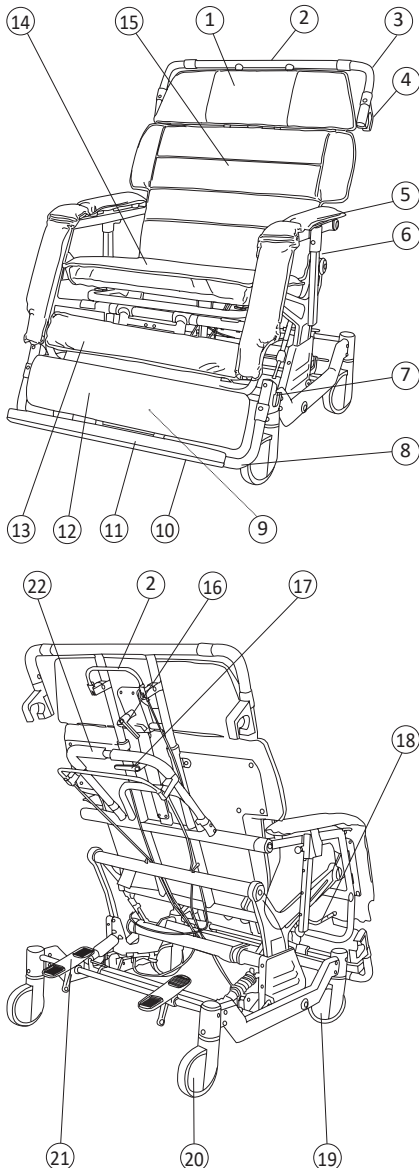
The equipment tilts smoothly and stops in any desired position.

All height adjustment screws are securely tightened. (I-400/I-700).
Height adjust pedals and actuation works smoothly (up & down) and stays at any desired position. (I-400X)

Components

1. Headrest
2. Headrest Handle
3. Chair Bracket (Headrest)
4. Plastic Bearing
5. Armrest
6. Armrest Height Adjustment Knob
7. Plastic Bearing
8. Chair Bracket (Footrest)
9. Footrest Height Adjusting Lever (Back Side)
10. Footrest Guard
11. Footrest
12. Leg Rest
13. Calf Pad
14. Seat
15. Backrest
16. Headrest Depth/ Angle Adjusting Lever
17. Headrest Height Adjusting Lever
18. Leg Rest Angle Adjusting Lever
19. Front Steering Caster
20. Rear Caster
21. Caster Control Pedal
22. Push Handle


For I-400X (not visible in illustration):

23. Footrest height adjustable lever (front side)
24. Pedal for height adjustment, raising
25. Pedal for height adjustment, lowering


Safety Warnings

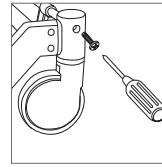
You must read and understand the warnings and cautions in this manual to use the Convertible I-400/I-700 positioning and transfer chair safely. Failure to observe them can lead to an accident and can seriously injure the patient or caregiver.

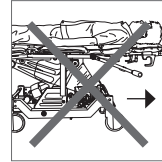
After any adjustment, repair or service and before use, ensure that all attaching hardware is tightened securely.

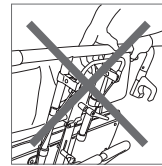
Do not attempt to move in fully reclined or nearly reclined position.

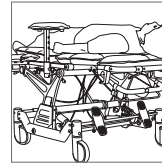
Do not push or pull the chair by the headrest handle.

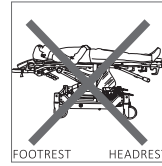To maintain maximum stability when reclining/ inclining, position the patient with arms and legs inside the chair and clear of moving parts.

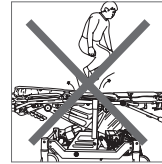
Do not confuse the headrest and footrest when placing the patient in the prone position.

Do not jump up and down on the back or seat cushion.

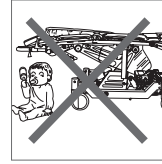
Do not use stairs or escalators to move an occupied chair in any position.


Ensure that all persons and objects are clear of the chair when you recline/incline the equipment.

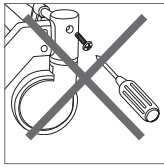
Never sit or lean on the headrest or leg rest when the chair is in the supine position.

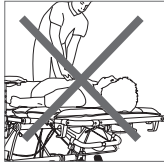
⚠ Never disassemble or remodel the Convertible chair. Use only parts and/or accessories authorized by the manufacturer for repair.

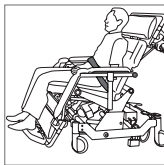
⚠ The Convertible chair is not designed to properly support a patient receiving CPR. This will change the center of gravity and may cause the equipment to tip over.

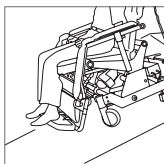
⚠ It must be determined prior to use by a licensed therapist or health care professional if a seat positioning strap must be used with the chair.

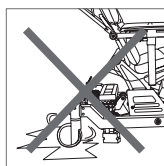
⚠ Take special care when going up and down a ramp. Never park the chair on a slope. Only operate chair on handicap grade access ramp.

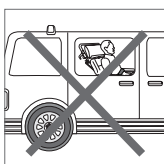
⚠ Do not use on gravel, frozen roads or any surface that is extremely slick or wet.

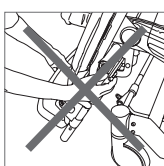
⚠ This equipment has no tie down system approved for transporting an occupied chair in a moving vehicle of any type. Do not use for transport in vehicles of any kind while the equipment is occupied.

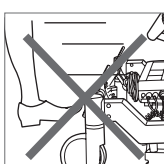
⚠ Do not attempt to lift the chair by holding the leg rest assembly.

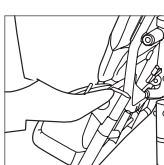⚠ Do not attempt to use the caster locks to stop the Convertible Chair in motion. These locks are not designed to be braking devices.

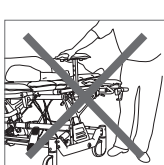
⚠ Do not use the footrest as a step when you sit on or get off the Convertible Chair.


⚠ Do not attempt to move the Convertible Chair by holding the armrest.

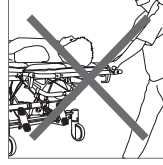
⚠ The Convertible Chair is designed for a single occupant. Never allow two or more people to rest on the chair, nor allow any heavy objects to be placed on the chair.

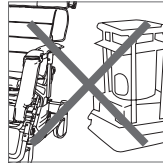
⚠ The Convertible Chair is not designed to be used as a stretcher, weight training or rehabilitation apparatus. Never use it for such purposes.

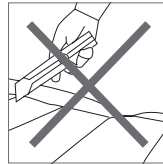
⚠ Do not use near an open flame (stove, cigarette, etc.) or an extreme heat source.

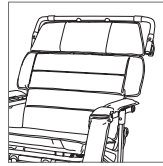
⚠ Never apply cutters or sharp edged objects to any of the cushions.

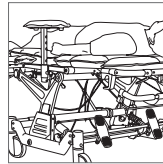⚠ Do not use the Convertible Chair with any missing or abnormal parts.

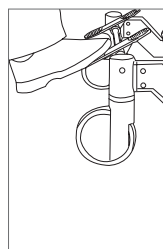⚠ Always raise the armrests when the patient is in the supine position. The armrests act as side rails.

⚠ Take special care to lock the casters securely. Ensure that all casters are locked when raising or lowering the patient (I-400X).

Especially for I-400X, in case of forward tilt:

- Ensure that patient is secured with belt or 2nd caregiver before using forward tilt.
- Ensure that chair is fully lowered and footrest fully retracted before using forward tilt

For height adjustment:

- Use only on level surfaces when chair is fully raised.
- Keep hands and feet clear of frame area when raising/lowering.

⚠ Take note of the maximum user weight. People who weigh more than the maximum user weight cannot use these models.

I-400/I-400X MAX 400lbs/181kg

I-700 MAX 700lbs/317.5kg

Product Label and Symbols

The product label contains information about the product.

Sample label:

GGGGGG			
REF EEEEEE	YYYY-MM	HHHkg/DDDlbs	
UPC F FFFFF FFFFF F	Human Care HC AB Årstaängsvägen 21B SE-117 43 Stockholm		
US Patent Numbers 5,819,339 and 5,697,109 Other US and Foreign Patents pending		Made in China by Human Care	
SN AAAAABBCDDDD			Label 90215

www.humancaregroup.com

	CE Marking
	Date of manufacture
	Manufactured for Human Care
	Maximum load
	Instructions: Read the instructions and make sure you understand them before use
REF	Part number/reference number
SN	Serial number
	Warning
	The device contains electronics and it must be recycled appropriately
	Applied part type B: approved for physical contact with patients (IEC 60601-1)
MD	Medical Device

Usage Guidelines

Adjusting the Convertible Chair Seat Height

Seat Height Adjustment

The Convertible Chair I-400/I-700 is adjustable within a 4"/10cm range. Do not set height above 4"/10cm mark.

For I-400X:

- Use foot to fold down height adjust pedal
- Use foot to pump pedal until desired height is reached
- Use foot to press lowering lever
- Fold pedal up when not in use

- With a 5mm Allen Wrench loosen the six bolts on the height adjustment diecasts on both the left and right side. Be careful not to unscrew the bolts.
- Using the Allen Wrench, rotate the bolt counter-clockwise to raise and clockwise to lower alternating between left and right sides in 1/2"/1.3cm increments. Repeat until seat height reaches desired height.
- Tighten bolts ① and ② on both sides. After tightening these bolts, tighten bolt ③ on both sides.

Make sure all bolts on both sides are fully tightened after finishing height adjustment of seat.

Convertible Chair Levers

The Convertible Chair has three adjusting levers:

- Headrest height adjusting lever
- Headrest depth/angle adjusting lever
- Footrest height adjusting lever

Warning: I-400X footrest is non-adjustable!

These levers which are in the same form have two functions:

Function 1: Tighten / Loosen;
Function 2: Change lever position in a tightened state

Never change the lever position in a loosened state - the lever cannot be adjusted unless it is tightened.


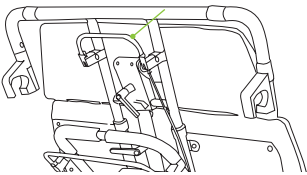
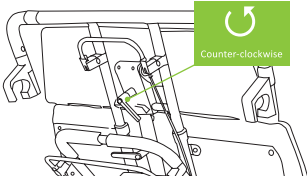
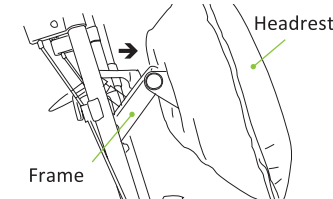
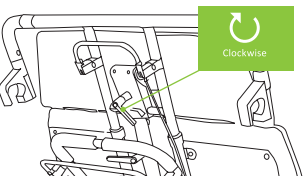
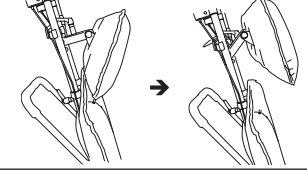
Lever Function 1





To tighten lever for adjustment turn clockwise. When tightened, make adjustment according to the instructions for applicable lever. To loosen lever turn counter-clockwise.

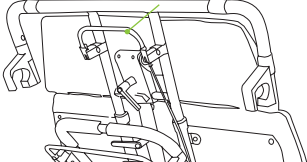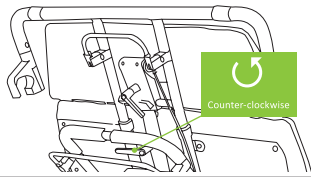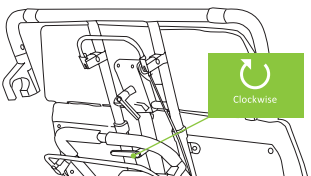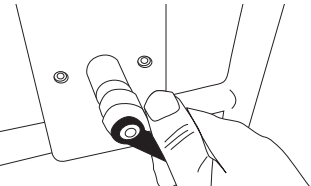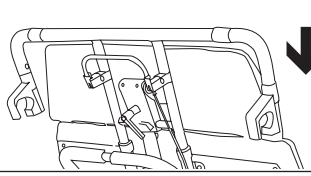Lever Function 2

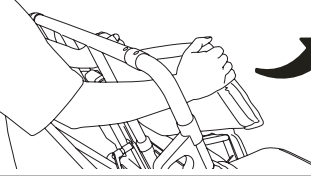
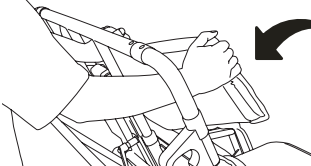
Adjust the lever position by pulling lever out while in a tightened state.

Adjusting the Convertible Chair Headrest

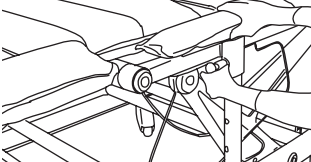
<p>Depth/Angle Adjustment</p> <p> When loosening the headrest depth/angle adjustment lever, be sure to hold the headrest securely in place. Loosening the lever with the headrest unsupported may cause the headrest to drop abruptly and an injury or accident may occur.</p>	
1. Hold the headrest handle securely with one hand.	
2. Loosen the headrest depth/angle adjusting lever with the other hand by turning it counter-clockwise.	
3. Position by pushing out the headrest (both depth and angle) when the adjusting lever is in its loose position.	
4. Once the depth and angle are correctly positioned, secure the headrest in place by turning the lever clockwise.	
5. To return the headrest to its original angle, hold the headrest securely in one hand so that it does not fall abruptly and loosen the lever by turning it counter-clockwise.	
6. Move the headrest from the existing position to the original position and secure in place by turning the adjustment lever clockwise.	
<p> When moving the headrest back to the home position, ensure that nothing comes between the headrest and the chair frame.</p>	
<p>Raising/Lowering the Headrest</p> <p> Always use both hands when raising/lowering headrest otherwise an injury or accident may occur.</p>	
1. To raise the headrest, first hold the headrest handle with one hand.	

2. Loosen the headrest height adjusting lever with the other hand by turning it counter-clockwise	
3. Raise the headrest slowly. When the headrest reaches the desired height, tighten the lever fully clockwise. Note: At the correct height, the patient's neck is visible.	
4. To return the headrest height to its original position, hold the headrest handle with one hand and loosen the height adjustment lever with the other hand.	
5. Once loosened, lower the headrest slowly to its lowest position and tighten the lever fully.	
<p> IMPORTANT! Be sure to restore the headrest to the lowest position before transforming the Convertible Chair to the supine position otherwise the patient's head could get caught between the back and the headrest resulting in injury or accident.</p>	




Adjusting the Convertible Chair Head Support

<p> When moving the headrest back to the home position, ensure that nothing comes between the headrest and the chair frame.</p>	
1. Hold the middle part of the headrest with one hand and turn one side at a time slowly inward with the other hand.	
2. To return the headrest to its original position, hold the middle part with one hand and pull one side at a time slowly towards you with the other.	

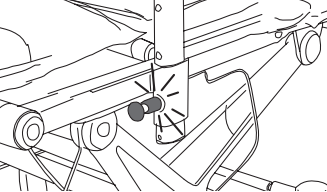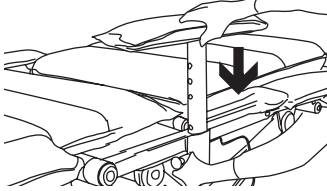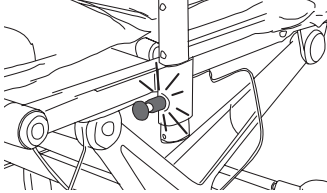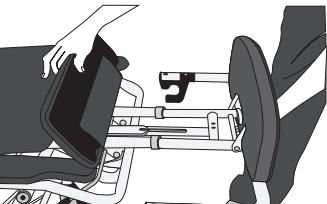
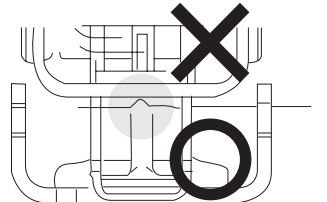
Adjusting the Convertible Chair Armrests

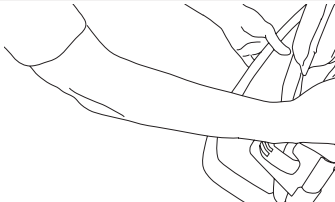
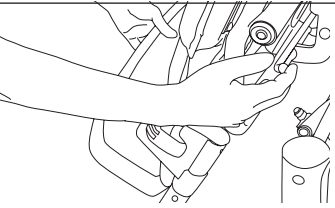
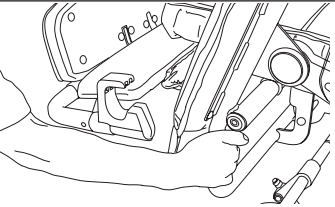
<p>The armrests can be used as siderails by raising them.</p>	
<p> Warning: Ensure that armrests are securely locked in place. An unsecured armrest may fall abruptly and an injury or accident may occur.</p>	
1. Pull the armrest height adjustment knob with one hand while simultaneously raising the armrest with the other hand.	

Usage Guidelines

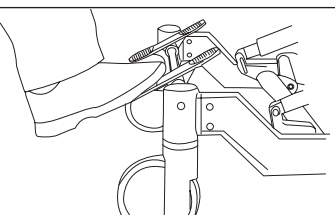
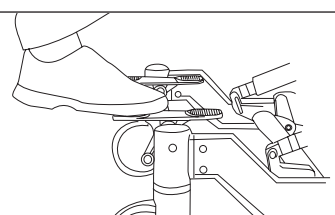
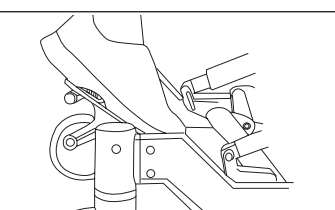
<p>2. Once the side armrest is at the desired height, release the height adjusting knob and the armrest will lock into place.</p>	
<p>3. To lower the armrest, pull the armrest height adjustment knob with one hand while lowering the armrest with the other.</p>	
<p>4. Once the side armrest is at the desired height, release the height adjusting knob and the armrest will lock into place.</p>	

Adjusting the Convertible Chair Footrest

<p>Adjusting the Height</p> <p></p> <p>Use both hands when raising or lowering the footrest otherwise injury or accident may occur. I400x: Lever for adjustment is located at the front of the chair.</p>	
<p>1. Hold the footrest guard handle with one hand and loosen the footrest height adjusting lever with the other.</p>	
<p>2. Raise or lower footrest slowly. When it reaches the desired height, tighten the lever fully. Caution: Position the footrest as shown - with the lever up, the leg rest may not be adjusted properly.</p>	
<p>Adjusting the Angle</p> <p></p> <p>Never use the footrest at any other angle than 0 degrees or 90 degrees. Using the footrest at any angle between 0 and 90 degrees may cause an injury or an accident as the footrest may suddenly fold.</p>	
<p>1. Use the footrest only at 90 degrees when the chair is in the upright position and only at 0 degrees when in the supine position.</p>	

<p>Adjusting the Leg Rest</p> <p></p> <p>Always use both hands when raising or lowering the leg rest. Do not snap the lever - snapping may lead to mechanical problems.</p>	
<p>1. Hold the footrest guard handle with one hand.</p>	
<p>2. Raise the leg rest slowly by holding and pulling the leg rest adjusting lever and guard handle with the other hand.</p>	
<p>3. To lower the leg rest, hold the footrest guard handle with one hand. Lower the leg rest slowly by holding and pulling the leg rest angle adjusting lever and guard handle together with the other hand.</p>	

Adjusting the Convertible Chair Casters

<p>The Convertible Chair has three caster functions:</p> <ol style="list-style-type: none"> 1. Lock Function 2. Free Function 3. Steering Lock Function <p></p> <p>The caster lock is not designed to stop the Convertible Chair in motion. Do not operate the caster control panel while moving the equipment.</p>	
<p>Caster Lock Function</p>	
<p>1. Use this function to hold the Convertible Chair motionless when transforming it from a supine to an upright position and when transferring a patient to and from a bed. Step down towards you to initiate.</p>	
<p>Caster Free Function</p>	
<p>1. Use this function to bring the Convertible Chair alongside a bed or to "freewheel". This function allows all four wheels to swivel and rotate freely. Engage neutral position to initiate.</p>	
<p>Steering Lock Function</p>	
<p>1. Use this function to lock the two front casters from swiveling so that the Convertible Chair can easily be steered in a straight line. Push pedal forward to engage steer function.</p>	

Adjusting the Convertible Chair Brackets

The Convertible Chair brackets are installed on each side of the headrest and footrest.

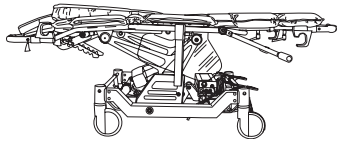
Use the chair brackets only when in the supine position. Protruding chair brackets present a great danger as people and objects may get caught on them. After transferring a patient, be sure to restore the chair brackets to their original position before transforming the equipment to the upright position.


Use the chair brackets only when transferring the patient using the transfer bar otherwise mechanical failure may result.

Adjusting the Chair Bracket


1. Bring the Convertible Chair to a supine position alongside the bed and lock the casters.

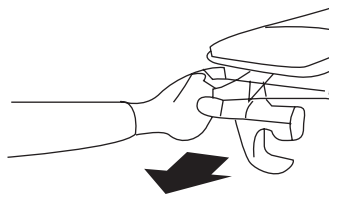
2. Hold the chair bracket opposite the bed side with one hand and the plastic ring on the guard handle with the other.

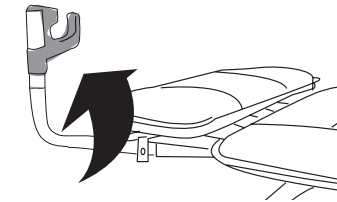
3. Turn the chair bracket 90 degrees while pulling the guard handle with the other hand.

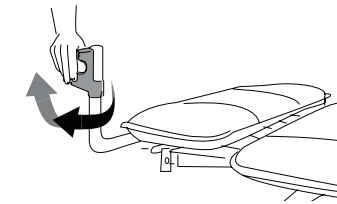
4. While holding the upright chair bracket with one hand and the plastic bearing with the other, pull and twist the chair bracket away from the bed and release it. Repeat for all brackets.

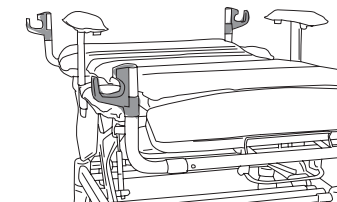5. When finished the chair brackets should be as shown and will be ready to install the PTS (Patient Transfer System).

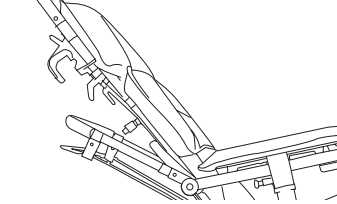6. To restore the chair to its original position, reverse above steps. Caution: When restoring the chair brackets to their original position, always ensure that the plastic bearings are down.

Operating the Convertible Chair

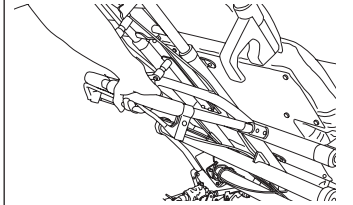
The Convertible Chair has unlimited positioning capabilities, including the Trendelenburg position.


Always use both hands when positioning the chair. Do not snap the levers - snapping may lead to mechanical problems.

Tilting the Convertible Chair

1. To tilt the chair forward or rearward, hold the headrest handle with one hand, squeeze the tilting lever and push both handles together. Push both slowly downwards or upwards.

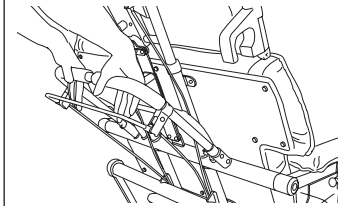
2. Release the tilting lever when the tilting reaches the desired angle.


Sitting to Supine Position

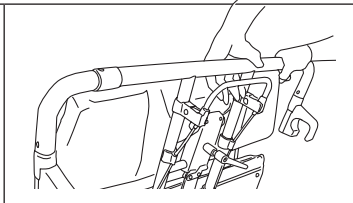1. To change from a sitting to the supine position, hold the headrest handle with one hand and squeeze the reclining lever on the headrest handle together with the other.

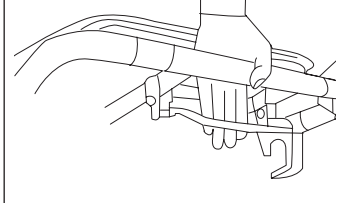
2. Push both slowly downward. Release the reclining lever when the back reaches the desired angle.


Supine to Sitting Position

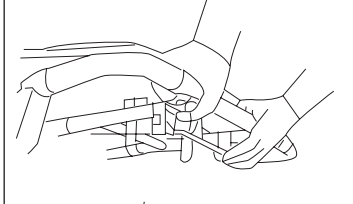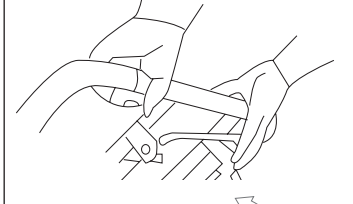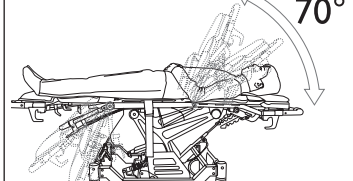
1. Hold the headrest handle with one hand and squeeze the reclining lever and headrest handle together with the other.

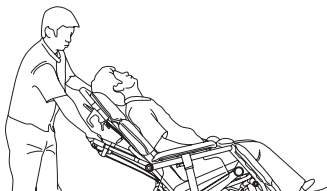
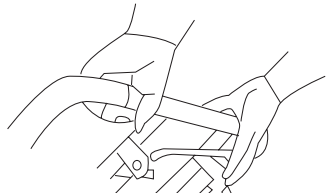
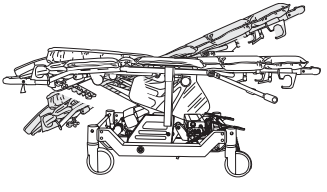
2. Lift slowly upward. Release the reclining lever when the back reaches the desired angle.


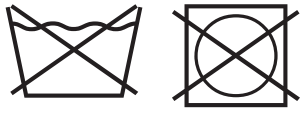
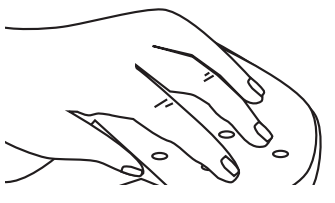
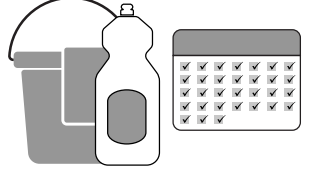
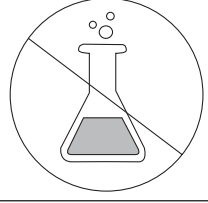
Note: Back adjustment angles can range between the supine position and 70 degrees.

Trendelenburg Position  It is recommended to secure patient in chair before positioning in Trendelenburg position. Ensure that the tilt position is in its most forward position before putting patient in Trendelenburg to prevent patient from sliding towards the ground and out of the chair.	
1. Hold the headrest handle with one hand and squeeze the tilting lever and push handle together with the other hand. Push both slowly downwards to full back tilt-in-space recline position.	
2. Hold the headrest handle with one hand and squeeze the reclining lever and headrest handle together with the other.	
3. Pull down to Trendelenburg position.	
 Lower patient slowly and deliberately to prevent patient from sliding back towards the caregiver and to the ground when going into Trendelenburg position.	

Cleaning Recommendations

 The materials used to manufacture SOFlux LVT® and the performance of the fabric may be adversely affected by permitting any cleaning agent to remain in contact with the fabric for a prolonged duration or a failure to adequately rinse the agent from the fabric. Before using any cleaning product, test it on an inconspicuous area of fabric.	
Routine Care	
1. Do not use harsh cleaners or solvents. Wipe fabric clean with a neutral soap and lukewarm water. Machine laundering or drying is not recommended. Do not pressure wash.	
2. After washing, wipe over with Sodium Dischloroisocyanurate (NaDCC) at 1,000ppm of available chlorine (or other chlorine releasing agent). Alternatively wipe over with a 70% Isopropyl Alcohol wipe. Dry thoroughly.	

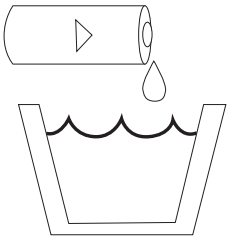3. For hard to clean areas, soft sponge with a liquid cleaner as specified on the manufacturer's product label.	
4. To control or prevent odors on long term incontinent applications, clean daily. The use of a scented cleaner / disinfectant is recommended.	
5. Do not use Hypocarbonate or Phenol based cleaning agents, any volatile liquids such as thinners or petroleum, or any abrasive compounds or pads when cleaning the chair.	
Disinfection  Iodophor type disinfectants (for example, Betadine) will stain fabric.	
1. Most phenolic or quaternary type disinfectants can be used. Dilute in accordance with manufacturer's instructions. Note: Undiluted Dettol may damage paint and plastic parts if prolonged contact is permitted.	
2. Gloves and protective clothing should always be worn when carrying out cleaning procedures in accordance with facility guidelines.	
 Important! When equipment has been used by a patient known or suspected of having an infection, this equipment must be disinfected before subsequent usage.	
Blood Contamination	
1. SOFluxLVT® can be disinfected with a 1:10 dilution of household bleach (5.25% sodium hypochlorite) as recommended by the CDC ¹ . Weaker dilutions (example 1:100) may be used, but may not be in accordance with CDC recommendations.	

¹ US Department of Health and Human Services (February 1989)

Material

Fabric information is offered as being typical for the indicated product by the manufacturer of the fabric. The values are based on laboratory results and are offered for guidance only. Lot to lot variations may occur that are not material to the performance of the product.

Outside Fabric

SOFlux LVT (low vapor transmission) is a 6.75oz/sq. yard 70 Denier/210 thread count nylon fabric with a blended neoprene/butyl rubber coating. The product is designed for applications where a severe reduction of the Moisture Vapor Transmission Rate (MVT) is desired for the benefit of infection control.

I-400X: Herculite Sure-Chek Fusion III HP

Outside Fabric Technical Data

EPA Registration Bacteriostat	10466-28
EPA Registration Fungistat	10466-23
Antibacterial	AATCC Method 147-1988 Saphylococcus aureus Contact inhibition 99%+
Antifungal	AATCC Method 30-1988 Saphylococcus aureus Contact inhibition 99%+
Primary Skin Irritation	Draize Dermal Toxicity, Non-Allergenic
Flame Resistance	NFPA 701 LS - Pass CAL 117 - Pass
Cigarette Ignition	FF4-72/16CFR, Part 1632 Pass (as a component in a finished mattress)
Moisture Vapor Transmission	ASTM E96/Method A .5g/m2/hr
Hydrostatic Burst (PSI)	Federal Standard 191-5512, 135lbs/61.2kg
Tear Strength	Federal Standard 191-5134 Warp, 4.6 lbs/2.1kg
Break Strength	Federal Standard 191-5100 Warp, 130 lbs/in (59kg/2.54cm) Fill, 105 lbs/in (47.6kg/2.54cm)
Abrasion Resistance	ASTM D 3886-92, 1300 cycles

Interior Fabric

Cushion material is a urethane foam, which is the same material used in most hospital mattresses. Not only does it prevent bacterial growth, but is also not susceptible to damage from short term wet conditions. This material is also ideal for preventing pressure ulcers from extended immobility.

Frame

The frame is composed of steel tubing and aluminum and plastic parts. Ensure all moisture is wiped off parts to avoid rusting and noise.

Accessories

Part number	Product name
B2001C	PTS kit for I-400 and I-330
B2011A	PTS kit I-400 Acute Care
B2013A	PTS kit I-700 Acute Care
B2020C	PTS kit I-700 Home Care
IVP-CHAIRS	IV Pole Convertible Chairs
B1081	Anti tip set

Troubleshooting

Check the following		
Problem	Check	Correction
Does not move	Caster lock	Release the caster lock (p.10)
Does not go straight	Position of caster control panel	Use the steering lock function (p. 10)
Headrest will not lock	Lever position	Correct the lever position (p.8,9)
Angle of leg rest not adjusted	Lever position	Adjust leg rest (p.10)
Forward tilt is not working forward or back to neutral position (I-400X)	Footrest is not hitting the floor. Mechanical locks are working properly. Gas pistons can slide freely in their slots. Gas pistons are working properly.	Adjust footrest to highest position (P.10). Adjust cable tension of mechanical locks or replace. Re-install gas piston mounting hardware. Replace gas piston and hardware.
Height adjustment is not working (I-400X)	Check for mechanical obstructions. Loose height adj. pedal linkage. Hydraulic cylinder is working properly.	Remove any mechanical obstructions. Tighten any loose hardware to remove free-play, replace if missing. Replace hydraulic cylinder.

Recycling

Once the product has reached the end of its life, it should be disposed according to your country's rules, especially for the separate collection of electrical and electronic products.

Correct disposal helps prevent negative consequences for the environment and human health.

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