

The Amputee Sling is a shoulder height model with divided leg parts. It features a reinforced back and is suitable for patients with double-sided or single leg amputations, patients with hip problems or patients with increased risk of sliding out of the sling. The Amputee sling is available in Polyester or Polyester Net fabrics.

#### Versatile

The Amputee Sling in Polyester fabric allows for fast and easy sling application making this sling ideal for most lifting situations. The Polyester Net fabric allows the sling to be used in a bathing or showering environment as well as for rehabilitation pools. A user of this sling must have good head control. The slightly reclined position helps prevent the patient from sliding out of the sling.

#### Extra Support

The Amputee Sling features a reinforced back and to offer additional support throughout the back area for those users with poor trunk stability as well as reinforced legs. The Amputee Sling can be used with or without

an optional waist belt that is available in an assortment of sizes with a Velcro or buckle closure. The unique design offers the added benefit that the patient can have their arms inside or outside the sling, depending on their capabilities.

#### Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced straps and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.



## Accessories



### Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no. 21180.\*



### Waist Belts

Velcro or buckle enclosure. Available in Junior and Adult sizing. Velcro Part no. 25145-\*\* Buckle Part no. 25146-\*\*\*



### Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866



### Hanger Bars for 2-, 3- or 4-point Suspension

Part no. 57551, 55764 [2- or 3-Point - Quick Release], 55619 [4-Point - Attach to leg hook]

## Lifting Situations



### Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



### Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



### Use to/from the floor


Suitable for transfers from the floor.



### Use for showering


Only Polyester Net fabric can be used for this application.

## Care and Features

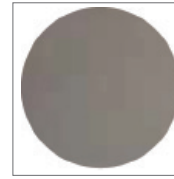


### Polyester

Durable, low-friction, easy to clean, apply and remove.

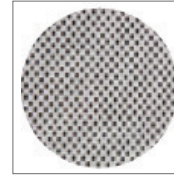

### Polyester Net

Soft, thin material that breathes and dries quickly. Ideal for bathing or showering.

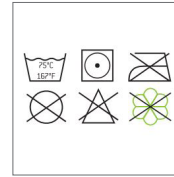

### Care


Care instructions located on sling label. Use a hospital grade laundry bag while laundering.



Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

## Technical Specifications

Part Number	Size	Colour	A	B	C
<b>Polyester</b>	<b>Polyester Net</b>				
25040-JM	25140-JM	Junior Medium	19cm / 7.4"	61cm / 24.1"	55cm / 21.7"
25040-S	25140-S	Small	26cm / 10.2"	72cm / 28.3"	57cm / 22.4"
25040-M	25140-M	Medium	36cm / 14.2"	78cm / 30.7"	59cm / 23.2"
25040-L	25140-L	Large	46cm / 18.1"	88cm / 34.6"	62cm / 24.4"
25040-XL	25140-XL	Extra Large	50cm / 19.6"	101cm / 39.8"	65cm / 25.6"
Maximum User Weight	300 kg / 660 lb				

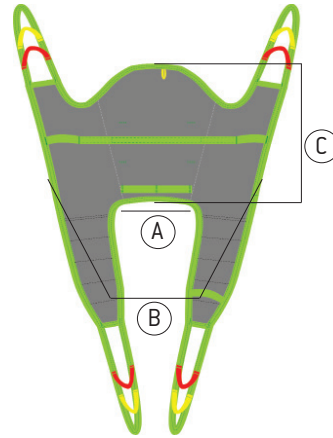

\* Available in assorted lengths: 10cm/3.9" (21180-10), 15cm/5.9" (21180-15), 20cm/7.8" (21180-20), 25cm/9.8" (21180-25), 35cm/13.7" (21180-35)

\*\* Junior Sizes: 40cm/15.7" (25145-40B), 50cm/19.7" (25145-50B), 60cm/23.6" (25145-60B); Adult Sizes: 60cm/23.6"/(25145-60), 70cm/27.6" (25145-70), 80cm/31.5" (25145-80), 90cm/35.4" (25145-90), 110cm/43.3" (25145-110), 120cm/47.3" (25145-120)

\*\*\* Junior Sizes: 40cm/15.7" (25146-40B), 50cm/19.7" (25146-50B), 60cm/23.6" (25146-60B); Adult Sizes: 60cm/23.6"/(25146-60), 70cm/27.6" (25146-70), 80cm/31.5" (25146-80), 90cm/35.4" (25146-90), 100cm/39.4" (25146-100), 110cm/43.3" (25146-110), 120cm/47.3" (25146-120)

This product is manufactured in accordance with all relevant ISO standards.



Learn More About This Product

