
Multi Sling



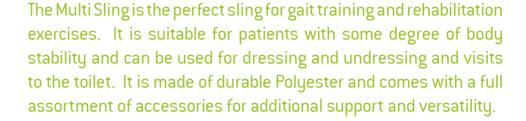








Multi Use

The Multi Sling is designed to act as support while doing standing, walking and balancing exercises, however, it can also be used for dressing and undressing and visits to the toilet. It consists of different parts - a chest belt, groin support and shoulder support. The Multi Sling can be used with either 2-point, 3-point or 4-point suspension and the user of this sling must have partial use of their legs.

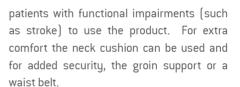



The Multi Sling offers a full range of optional accessories that increase its functionality. The chest belt can be used as a support during walking and balancing exercises and, when combined with the hip belt, it allows

Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. They feature reinforced straps and high strength nylon webbing around the sling to make them strong enough to resist even the most active use. Each sling label is colour coded to ensure quick and easy selection of the correct sling and the straps are colour coded for easy determination of required positioning.



HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B 117 43 Stockholm Phone:- +46 8 665 35 00 Fax:- +46 8 665 35 10 info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road Ottawa, ON K2E 7K1 Phone: +613.723.6734 Fax: +613.723.1058 info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124 8076 PA Vierhouten Phone: +31 577 412 171 Fax: +31 577 412 170 info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K Austin, TX 78754 Phone: +512.476.7199 Fax: +512.476.7190 info.us@humancaregroup.com

Accessories



Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no. 21180-*



Neck Cushion

For added comfort and support for the neck. Part no. 25150



Waist or Hip Belt

Waist belt with Velcro (Part no. 25145**) or with Safety Lock (Part no. 25146***). Hip belt Part no. 25107.



Upholstered Cover

Protects user from hanger bar impact. Not for use in a wet environment. Part no. 50866

Lifting Situations



Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period



Use for gait training/ mobilization

Suitable for gait training and rehabilitation exercises.



Use for dressing/ undressing

Suitable for dressing/ undressing applications.

Care and Features



Polyester

Durable, low-friction, easy to clean, apply and remove.



Components

Part no. 25106 Chest Belt. 25108 Groin Support, 25109 Shoulder Support



Care instructions located on sling label. Use a hospital grade laundry bag while laundering.



Technical Specifications

Part Number	Size	Colour	Weight Range	Waist Range	A	В	С	
Polyester			(Recommended)	(Min-Max)				
25105-S	Small	S	30kg -60kg / 66.1 lb - 132.3 lb	70cm - 86cm / 27.6" - 33.9"	86cm/33.9"	36cm/14.2"	20cm/7.9"	
25105-M	Medium	M	55kg -90kg / 121.3 lb - 198.4 lb	86cm - 104cm / 33.9" - 40.9"	104cm/40.9"	38cm/15"	20cm/7.9"	
25105-L	Large	L	85kg - 120kg/ 187.4 lb - 264.6 lb	104cm - 122cm /40.9" - 48"	122cm/48"	40cm/15.8"	20cm/7.9"	
25105-XL	Extra Large	XL	120kg - 150kg / 264.6 lb - 330.7 lb	122cm - 140cm / 48" - 55.1"	140cm / 55.1"	42cm/16.5"	20cm/7.9"	
Maximum User Weight		200kg/440.1lb						

^{*} Available in assorted lengths: 10 cm/3.9 " (21180.10), 15 cm/5.9 " (21180.15), 20 cm/7.8 " (21180.20), 25 cm/9.8 " (21180.25), 35 cm/13.7 " (21180.35)** Available Sizes: 60 cm/23.6 " (25145.60), 70 cm/27.6 " (25145.70), 80 cm/31.5 " (25145.80), 90 cm/35.4 " (25145.90), 110 cm/43.3 " (25145.110),

¹²⁰cm/47.3" [25145-120]

*** Available Sizes: 60cm/23.6"/(25146-60), 70cm/27.6" (25146-70), 80cm/31.5" [25146-80), 90cm/35.4" (25146-90), 100cm/39.4" (25146-100), 110cm/43.3" (25146-110), 120cm/47.3" (25146-120)