




Fitting Protocol

Highback Plusline Sling
Model 25120

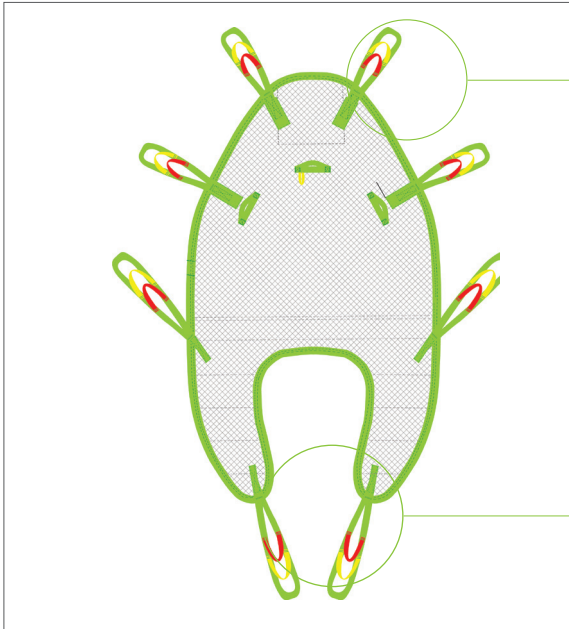


- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.

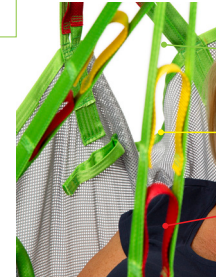

- Green Strap**
Increased Recline
Less hip flex and greatest angle
- Yellow Strap**
Semi-Recumbant
Mid-hip angle
- Red Strap**
Upright Position
Least hip angle

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 318kg/701lb

Model 25120 - Polyester/Polyester Net

Transfer Situation:

- To/From a Bed
- To/From a Chair
- To/From the Floor

Leg Placement:

- Divided relaxed position
- Open position
- Divided position
- Closed position

Suspension:

- 2-Point
 - 4-Point
 - 3-Point
- Not available for this sling**

Available sizes:

Indicate size required

- S
- M
- L
- XL

Accessories required:

- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large
- 25102 Reinforced Leg Support - One Size
- 50866 Upholstered Cover for Hanger Bar. **Do not use in wet environments.**

Patient Information:

Name	
Facility	Room
Lift Used	Hanger Bar Used

HUMAN CARE SWEDEN (HQ)
 Årstaängsvägen 21B
 117 43 Stockholm
 Phone:- +46 8 665 35 00
 Fax:- +46 8 665 35 10
 info.se@humancaregroup.com

HUMAN CARE CANADA
 10-155 Colonnade Road
 Ottawa, ON K2E 7K1
 Phone:- +613.723.6734
 Fax:- +613.723.1058
 info.ca@humancaregroup.com

HUMAN CARE NEDERLAND
 Elspeterweg 124
 8076 PA Vierhouten
 Phone:- +31 577 412 171
 Fax:- +31 577 412 170
 info.nl@humancaregroup.com

HUMAN CARE UNITED STATES
 8006 Cameron Road, Suite K
 Austin, TX 78754
 Phone:- +512.476.7199
 Fax:- +512.476.7190
 info.us@humancaregroup.com

The Highback Plusline Sling is a high back model with reinforced and divided leg parts. It has a generous overall width to offer a stretched seating position that is well-suited for heavier patients. The reclined seating offers a balanced weight distribution and the combined Polyester/Polyester Net fabric makes it easy to apply. It features a reinforced head support is available in a Polyester/ Polyester Net combination that is well suited for most lifting requirements.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name		
Date	Position/Title	



Important! This fitting protocol does not replace the user manual. Read all instructions before use!