




# Fitting Protocol

Sit Sling  
Model 25030

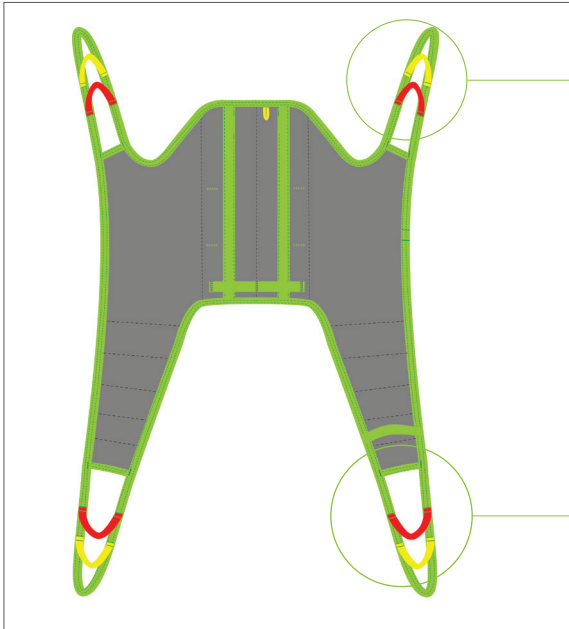

- Green Strap
- Yellow Strap
- Red Strap

**Extension Loops**


- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

- Green Strap
- Yellow Strap
- Red Strap

**Extension Loops**

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.

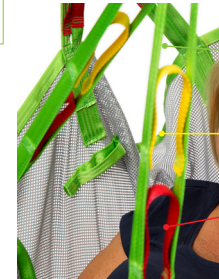


- Green Strap**  
Increased Recline  
Less hip flex and greatest angle
- Yellow Strap**  
Semi-Recumbent  
Mid-hip angle
- Red Strap**  
Upright Position  
Least hip angle

Match colours on both sides of the patient for a 90 degree angle in the hips.


**Maximum User Weight: 300kg/660lb**

**Model 25030 - Polyester**


**Transfer Situation:**

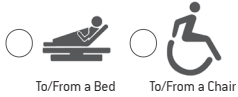

**Leg Placement:**

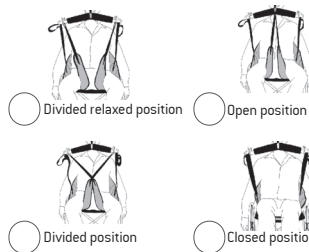

**Suspension:**

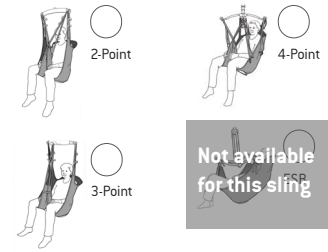

Available sizes:  Indicate size required



**Accessories required:**

- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large
- 25102 Reinforced Leg Support - One Size
- 50866 Upholstered Cover for Hanger Bar
- 25145.\* Waist Belt [Velcro]
 

<b>*Junior Sizes</b>	<b>*Adult Sizes</b>
40B (40cm/15.7")	60 (60cm/23.6")
50B (50cm/19.7")	70 (70cm/27.6")
60B (60cm/23.6")	80 (80cm/31.5")
	90 (90cm/34.4")
- 25146.\* Waist Belt [Safety Buckle]
 

<b>Size (eg. -60B)</b>	
<input type="text"/>	
	<b>*only for Model #25146</b>

**Patient Information:**

Name		
Facility	Room	
Lift Used	Hanger Bar Used	

**HUMAN CARE SWEDEN (HQ)**  
Årstaängsvägen 21B  
117 43 Stockholm  
Phone:- +46 8 665 35 00  
Fax:- +46 8 665 35 10  
info.se@humancaregroup.com

**HUMAN CARE CANADA**  
10-155 Colonnade Road  
Ottawa, ON K2E 7K1  
Phone: +613.723.6734  
Fax: +613.723.1058  
info.ca@humancaregroup.com

**HUMAN CARE NEDERLAND**  
Elspeterweg 124  
8076 PA Vierhouten  
Phone: +31 577 412 171  
Fax: +31 577 412 170  
info.nl@humancaregroup.com

**HUMAN CARE UNITED STATES**  
8006 Cameron Road, Suite K  
Austin, TX 78754  
Phone: +512.476.7199  
Fax: +512.476.7190  
info.us@humancaregroup.com

The Sit Sling is a shoulder high model with divided and reinforced leg parts as well as a reinforced back. It is designed to give the patient extra support and is a good solution for those users with poor trunk stability. Made of Polyester fabric, it is fast and simple to apply and can be used for most lifting situations. The Sit Sling creates an upright seating position which makes it ideal for proper positioning when transferring into a chair. The Sit Sling can be used with or without a back belt.

### Additional Instructions

To staff/caregiver:


To patient:


Fitting Protocol Completed by:

Name		
Date	Position/Title	



**Important!** This fitting protocol does not replace the user manual. Read all instructions before use!