




## Fitting Protocol

Bath Sling  
Model 25035

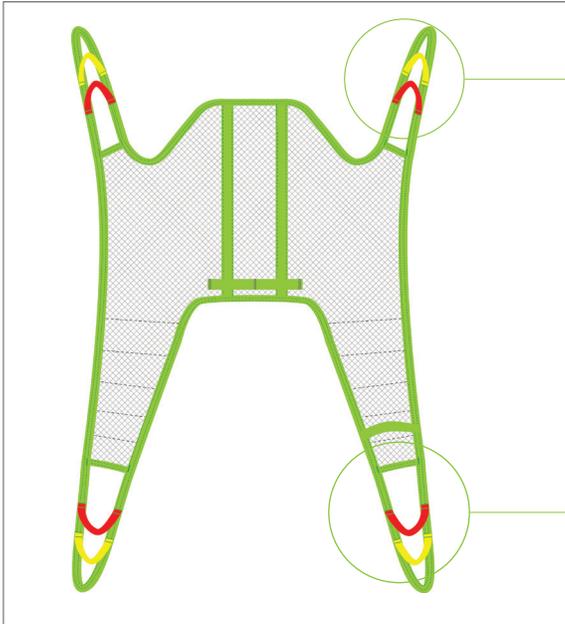

- Green Strap
- Yellow Strap
- Red Strap

### Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

- Green Strap
- Yellow Strap
- Red Strap

### Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.



- Green Strap**  
Increased Recline  
Less hip flex and greatest angle
- Yellow Strap**  
Semi-Recumbent  
Mid-hip angle
- Red Strap**  
Upright Position  
Least hip angle

Match colours on both sides of the patient for a 90 degree angle in the hips.


Maximum User Weight: 300kg/660lb

Model 25035 - Polyester Net


### Transfer Situation:



### Leg Placement:

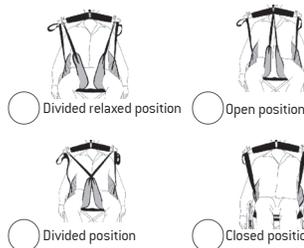

### Suspension:

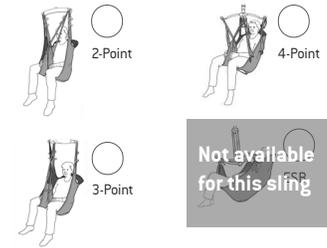

### Available sizes:

Indicate size required



### Accessories required:

- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large
- 25102 Reinforced Leg Support - One Size
- 25145.\*  
Waist Belt (Velcro)  

*Junior Sizes	*Adult Sizes
-40B (40cm/15.7")	-60 (60cm/23.6")
-50B (50cm/19.7")	-70 (70cm/27.6")
-60B (60cm/23.6")	-80 (80cm/31.5")
	-90 (90cm/34.4")
	-100 (100cm/39.4")
	-110 (110cm/43.3")
	-120 (120cm/47.3")
	-130 (130cm/51.2")
- 25146.\*  
Waist Belt (Safety Buckle)  

Size (eg. -60B)
<input type="text"/>

\*only for Model #25146

### Patient Information:

Name	
Facility	Room
Lift Used	Hanger Bar Used

**HUMAN CARE SWEDEN (HQ)**  
 Årstaängsvägen 21B  
 117 43 Stockholm  
 Phone: +46 8 665 35 00  
 Fax: +46 8 665 35 10  
 info.se@humancaregroup.com

**HUMAN CARE CANADA**  
 10-155 Colonnade Road  
 Ottawa, ON K2E 7K1  
 Phone: +613.723.6734  
 Fax: +613.723.1058  
 info.ca@humancaregroup.com

**HUMAN CARE NEDERLAND**  
 Elspeterweg 124  
 8076 PA Vierhouten  
 Phone: +31 577 412 171  
 Fax: +31 577 412 170  
 info.nl@humancaregroup.com

**HUMAN CARE UNITED STATES**  
 8006 Cameron Road, Suite K  
 Austin, TX 78754  
 Phone: +512.476.7199  
 Fax: +512.476.7190  
 info.us@humancaregroup.com

The Bath Sling can be used in most lifting situations and fits most patients but is most suitable for wet environments. Made of Polyester Net fabric, it dries quickly and can be left behind the patient for a short period of time after the transfer is complete, making it an ideal option for bathing or showering. It is a soft, comfortable material that can also be used for rehabilitation in pool areas. The Bath Sling creates an upright seating position and can be used with or without a back belt.

### Additional Instructions

To staff/caregiver:


To patient:


Fitting Protocol Completed by:

Name		
Date	Position/Title	



**Important!** This fitting protocol does not replace the user manual. Read all instructions before use!