




Fitting Protocol

Comfort Sling Soft &
Comfort Sling High Soft
Model 25064, 25069

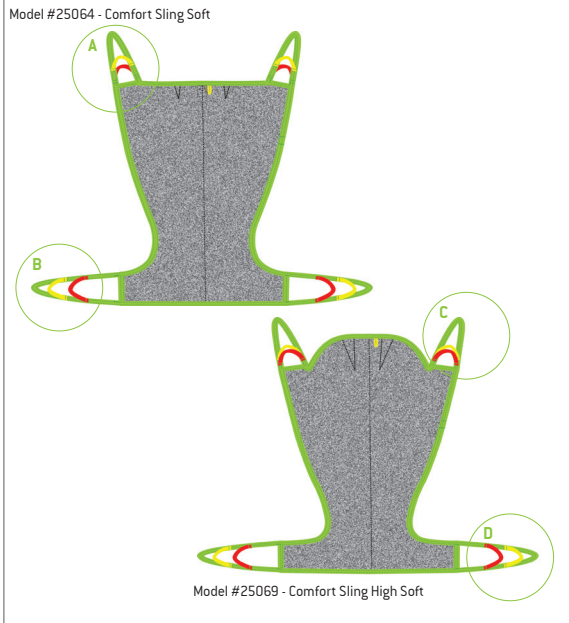

Sling Loop Letter:

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Sling Loop Letter:

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.



- Green Strap**
Increased Recline
Less hip flex and greatest angle
- Yellow Strap**
Semi-Recumbent
Mid-hip angle
- Red Strap**
Upright Position
Least hip angle


Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 300kg/660lb

Model 25064 - Soft Air Mesh

Model 25069 - Soft Air Mesh

Transfer Situation:

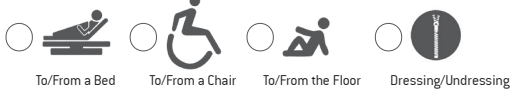

Available sizes:

Indicate size required



Leg Placement:



Divided relaxed position



Open position




Divided position



Closed position

Suspension:

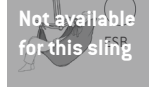

Accessories required:



- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large



- 25102 Reinforced Leg Support - One Size



- 50866 Upholstered Cover for Hanger Bar

Patient Information:

Name

Facility

Room

Lift Used

Hanger Bar Used

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone: +46 8 665 35 00
Fax: +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

Our Comfort Slings are designed to give the patient comfort and a sense of safety. The undivided legs evenly distribute the patients weight to reduce pressure and the Soft Air Mesh fabric promotes less repositioning making this sling a suitable choice for patients with pain. In addition, the material does not promote skin breakdown and can therefore be left behind the patient for a short periods of time. The Comfort Slings is available in two models, one shoulder height model and one with extra head support.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name		
Date	Position/Title	



Important! This fitting protocol does not replace the user manual. Read all instructions before use!