




Fitting Protocol

Amputee Sling
Model 25040, 25140

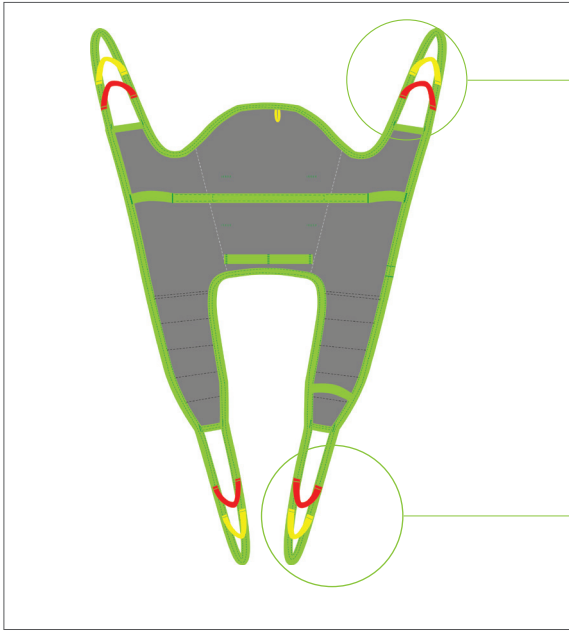


- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.

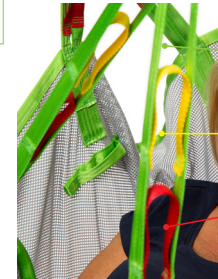

- Green Strap**
Increased Recline
Less hip flex and greatest angle
- Yellow Strap**
Semi-Recumbent
Mid-hip angle
- Red Strap**
Upright Position
Least hip angle

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 300kg / 660lb


Model 25040 - Polyester

Model 25140 - Polyester Net


Transfer Situation:



Leg Placement:

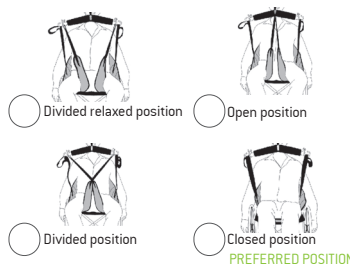

Suspension:

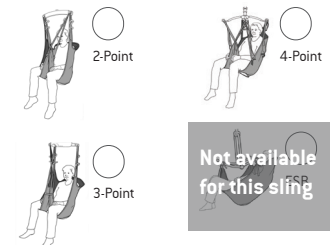

Available sizes:

Indicate size required



Accessories required:



- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large



- 25102 Reinforced Leg Support - One Size



- 50866 Upholstered Cover for Hanger Bar. **Do not use in wet environments.**



- 25145-*
Waist Belt
[Velcro]

*Junior Sizes *Adult Sizes
-40B (40cm/15.7") -60 (60cm/23.6")
-50B (50cm/19.7") -70 (70cm/27.6")
-60B (60cm/23.6") -80 (80cm/31.5")
 -90 (90cm/34.4")



- 25146-*
Waist Belt
[Safety Buckle]

Size (eg. -60B)
-100[†] (100cm/39.4")
-110 (110cm/43.3")
-120 (120cm/47.3")
-130 (130cm/51.2")
[†]only for Model #25146

Patient Information:

Name	
Facility	Room
Lift Used	Hanger Bar Used

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

The Amputee Sling is a low height model with divided leg parts. It features a reinforced back and is suitable for patients with double-sided or single leg amputations as well as patients with hip problems. Because of the slightly recline position, it is also suitable for patients with an increased risk of sliding out of the sling. The user of this sling must have good head control. The Amputee sling is available in Polyester or Polyester Net fabrics and can be used with an optional belt for added security.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name		
Date	Position/Title	



Important! This fitting protocol does not replace the user manual. Read all instructions before use!