




Fitting Protocol

Positioning Sling
Model NSB-401-ST, NSB-410-ST,
NSB-451-U

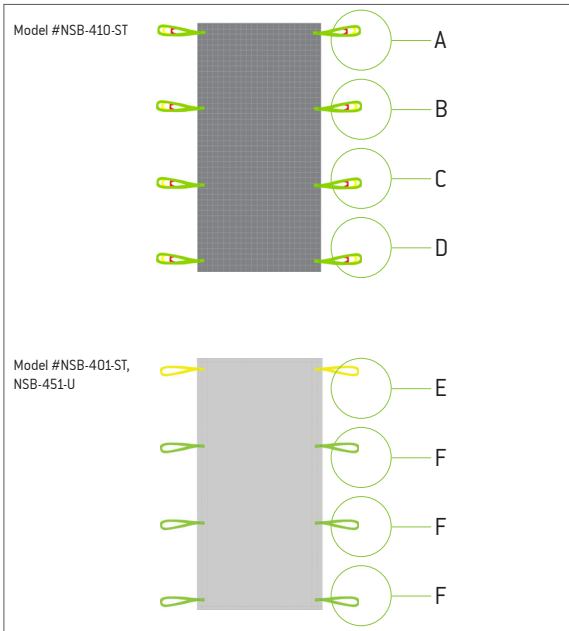

- Strap A**
- Red Strap
 - Yellow Strap
 - Green Strap

- Strap B**
- Red Strap
 - Yellow Strap
 - Green Strap

- Strap C**
- Red Strap
 - Yellow Strap
 - Green Strap

- Strap D**
- Red Strap
 - Yellow Strap
 - Green Strap

- Yellow Strap E
- Green Strap F

Extension Loop Size:

Strap A: _____

Strap B: _____

Strap C: _____

Strap D: _____

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.



- Green Strap**
Increased Recline
Less hip flex and greatest angle
- Yellow Strap**
Semi-Recumbent
Mid-hip angle
- Red Strap**
Upright Position
Least hip angle

Match colours on both sides of the patient for a 90 degree angle in the hips.

For Models NSB-401-ST and NSB-451-U angle adjustment not applicable.



- Yellow Strap**
Head positioning
- Green Strap**
Body positioning


Maximum User Weight: 454kg/1000lb

- Model NSB-401-ST, NSB-410-ST, Ventilated Polyester


Maximum User Weight: 318kg/700lb

- Model NSB-451-U - Single Patient Use Polyester/Cotton


Transfer Situation:

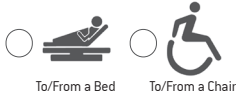


Available sizes:

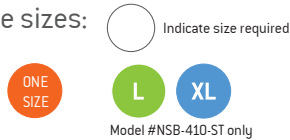

Leg Placement:

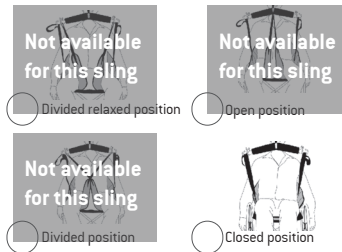

Suspension:

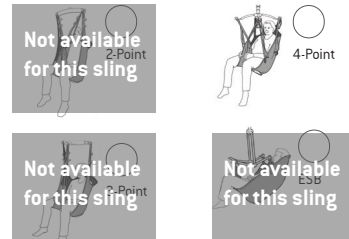

Accessories required:



Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Patient Information:

Name: _____

Facility: _____ Room: _____

Lift Used: _____ Hanger Bar Used: _____

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone: +46 8 665 35 00
Fax: +46 8 665 35 10
info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone: +613.723.6734
Fax: +613.723.1058
info.ca@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
info.nl@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K
Austin, TX 78754
Phone: +512.476.7199
Fax: +512.476.7190
info.us@humancaregroup.com

The Positioning Sling is designed to assist a caregiver to safely and securely perform safe patient handling during sudden or intended positioning/repositioning, turning and lateral transfer from a supine position. This sling is offered in a reusable or single patient use model. The reusable model can be placed behind the patient much like a sheet and can remain behind the patient for short periods of time. The unique material facilitates air flow and breathability and does not promote skin breakdown. The single patient use model offers the convenient option of disposal after use combined with the added benefit of hygiene control. All slings fit standard hospital beds.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name		
Date	Position/Title	



Important! This fitting protocol does not replace the user manual. Read all instructions before use!