



HUMAN CARE

Your Life. Your Way.



nexus


Assembly Instructions for Back Strap

Part no. 93310




Back Strap

Installing the Back Strap

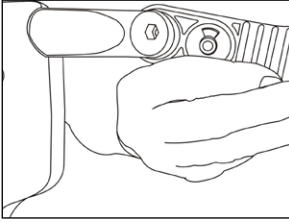

To install Back Strap, ensure key shape faces up. Place into clamp located at end of handle. Tighten into place with screw and nut.

Care and Maintenance

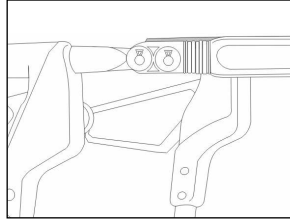

Periodically check to ensure screws are tight.



Recycling

Recycling of products

When recycling, the device's parts can be sorted according to material after unscrewing and dismantling rivets and other connections. The different parts should be separated into metal, plastic or combined materials.

Different authorities/recycling companies handle recycled materials in different ways. If in doubt about the best way to proceed with recycling, contact the relevant local governing body.

Environment friendliness

Human Care HC AB products are manufactured to have as little effect on the environment as possible during their lifetime. This allows for the used product to be dismantled and for its parts to be recycled. Products with lengthy life cycles safeguard our sensitive environment and are thus more environmentally friendly.

Human Care HC AB is a part of FTI.
FTI – A countrywide system for recycling of packaging.

Warranty

Human Care HC AB has high demands on all our product's quality and offers a warranty period against manufacturer's defects. Unless otherwise stated, the warranty period for these devices is one year, excluding wear and tear items (seat, hand grips). Contact your local office for complete warranty information or visit our website at www.humancaregroup.com.

Any complaints or questions can be made by contacting your local office or by completing a contact form at <https://humancare.se/contact-us/claim-management/>.

Europe: info.se@humancaregroup.com
Canada: info.ca@humancaregroup.com
United States: info.us@humancaregroup.com
Netherlands: info.nl@humancaregroup.com

Warnings



DO NOT USE the product if,



Weight capacity is exceeded.



Product is damaged in any way.



You have not read and understood all warnings and instructions.

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B

117 43 Stockholm

Phone:- +46 8 665 35 00

Fax:- +46 8 665 35 10

Email: info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road

Ottawa, ON K2E 7K1

Phone:- 613.723.6734

Fax:- 613.723.1058

Email: info.ca@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Street, Suite K

Austin, TX 78754 USA

Phone: 512.476.7199

Fax: 512.476.7190

Email: info.us@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124

8076 PA Vierhouten

Phone: +31 577 412 171

Fax: +31 577 412 170

Email: info.nl@humancaregroup.com

www.humancaregroup.com



HUMAN CARE
Your Life. Your Way.