

Limb Sling



Helpful
Comfortable
Secure



Our Limb Sling is designed to comfortably help lift, support and position a client's body part such as legs, arms, shoulders or hips. This provides better access to treatment, performing surgery, leg/vein dressing, foot care, wound care treatment, hygiene and more.

Comfortable design

The easy-care material is gentle on the skin and can work equally well on both the upper and lower extremities. Its padded design is comfortable to use and its flexibility allows it to contour to the body shape and angles needed for wound care.


example moving legs up in bed, behind the user's shoulders to help them lean forward.

Caregiver safety

Caregiver safety is an important issue in facilities. Using the Limb Sling reduces the caregiver's physical strain by reducing their bending and tilting movements as it acts as an extension of the arms while moving the patient's body part to the correct location and height.

Multi functional

With a cooperative client who has good trunk control and endurance, this harness can assist with turning, to raise a shoulder to access the head, or to raise the hips for an easy pelvic bridge. It is recommended that the Limb Sling be used with a ceiling lift or floor lift to promote caregiver safety and to avoid patient injury and/or skin friction. The harness can also be used for manual movements, for

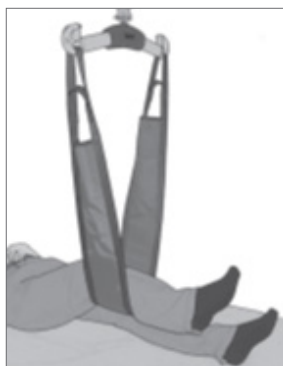

Accessories



Ceiling Lifts

Compatible for use with our Altair, Roomer S or HeliQ ceiling lifts



Floor Lifts

Compatible for use with most floor lifts, including our Diana II.

Lifting situations



Use for support

Suitable as support for one or both legs or one arm for such activities as wound treatment, dressing and hygiene.



Alternate support


Suitable as support for shoulders, pannus and hips for care related functions.



Use for mobilization


Suitable for mobilization exercises while in a bed.

Care

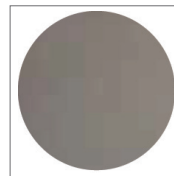


Material

Polyester med lågfriktionskoefficient och mjuk stoppning inuti för ökad komfort.

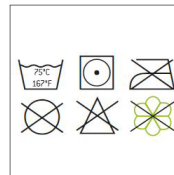

Care

Care instructions located on sling label. Use a hospital grade laundry bag while laundering.

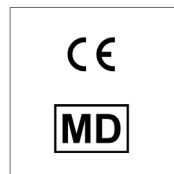

Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

Technical specifications

| Artikel-nummer | Storlek | A | B | C |
|----------------|---------|-------------|---------------|------------|
| 25700-M | OneSize | 122cm / 48" | 17,5cm / 6,9" | 33cm / 13" |

*We improve freedom of movement
and quality of care for people.*