
HUMAN CARE

Your Life. Your Way.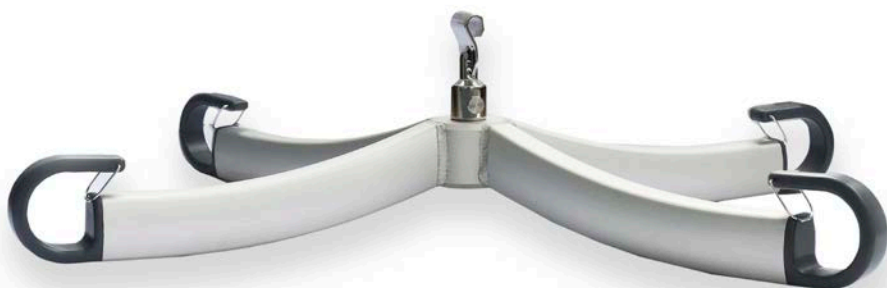

Bruksanvisning/User manual

Hanger Bar 4P HeliQ 92938

Innehåll

SVENSKA

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User Manual

Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

INFORMATION symbol indicates recommendations and information for proper, trouble-free usage.

WEIGHT CAPACITY symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

CE MARKING symbol indicates product meets requirements of EU Regulation 2017/745 (MDR).

MEDICAL DEVICE symbol indicates that the product is a medical device according to the definition of medical device in EU Regulation 2017/745 (MDR).

RECYCLE according to local regulations.

READ INSTRUCTIONS and ensure you fully understand them before using this product.

***Product modification is not permitted.
Before use, always check that the transfer sheet is not damaged or worn.***

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at www.humancaregroup.com.

Contact your local distributor if you have any questions about the product and its use.

Important instructions before use

Safety instructions

Before using the Hanger Bar for the first time, make sure that:

- You have read and understood the instructions for use of the Hanger Bar, the lift it is attached to and all of its accessories.
- The Hanger Bar has been installed in accordance with the user manual.
- The accessories which are intended for use with the Hanger Bar are correctly fitted.
- Before use, always check that the device is not damaged or worn. Especially the lift hooks should be free of any cracks.

Warning:

Product modification is not permitted.

Don't drop the hanger bar. If dropped, perform a visual inspection of the hanger bar and replace the complete hanger bar in case there are any cracks.

Important!


The Hanger Bar is designed exclusively for lifting people, and only one person at a time.

Preparations before use


Before and during use, always ensure that:

- You read and ensure you understand this user manual.
- Personnel, who use the equipment, have been given adequate training in the use of the Hanger Bar and its accessories.
- You should never leave the patient unattended during a lifting situation.
- The lift belt is not twisted or worn and that it can move freely in and out of the lift.
- Accessories, e.g. loop slings and hanger bar, are showing no signs of wear or other damages.
- The lift is correctly suspended in the distance strap, rail roller and rail.
- The right types, sizes, materials and designs of the accessories have been selected with respect to the patient's safety and needs.
- The accessories have been correctly installed as described in the manual. The sling's strap loops and suspension must be checked once the loops are fully extended when lifting but before the patient is lifted from the underlying surface.

- It is important to always check that the slings are at the bottom of all sling hooks securely placed below the locking pin (see picture below).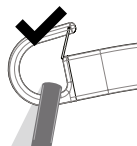

Correct

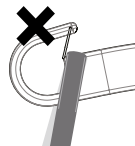In-correct

Important! If the maximum load differs between lift, Hanger Bar and slings, then the lowest maximum load shall always be used.


Important! To ensure that the lift belt will not be damaged, it is important that the Hanger Bar is balanced and that the sling is suspended correctly. Never hold or pull the lift belt and the Hanger Bar when moving a patient. It is also important that the lift belt is tensioned as it is wound in.

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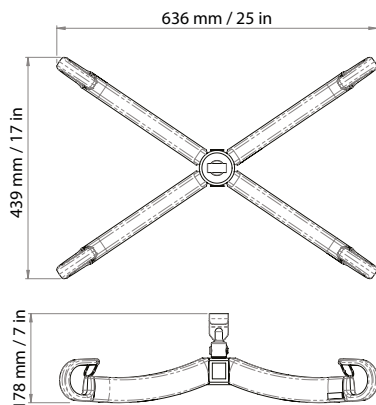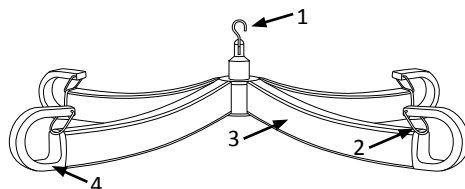
Intended use

The 4-point hanger bar creates more space within the sling which can improve the comfort for larger clients during the transfer process. This hanger bar is also suitable for transferring with a Positioning Sling, which is longer and has often 8 attachment loops. The dimensions and number of hooks of this hanger bar make it possible to use the Positioning Sling in its full extend.

The Hanger Bar is designed exclusively for lifting people, only one person at a time, and shall never be held during lifting. The Hanger Bar has been tested for indoor use and with Human Care loop slings only.

 Human Care provides suitable training in safe operation of the Hanger Bar and other accessories. Further information about slings and other accessories can be found at www.humancaregroup.com.

Human Care's products are continuously being developed and updated to provide our customers with the best products, we reserve the right to make design changes without prior notice.

Product label

The product labels contain information about the hanger bars. One label is located underneath the hanger bar.

Symbols on the unit and product label	
	Catalogue number / Part number
	Max. load
	Batch code symbol
	International article number
	Universal Product Code
	Manufacturing date
	Manufacturing address
	Read the user manual
	Medical Device mark
	According to European directive
	Don't drop the hanger bar
	Perform a visual inspection
	QR code to our webpage
	Particular attention is required

Compliances and Standards

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO 10535.

Hanger Bar 4P HeliQ


Attaching and detaching

Installation

The attachment shall only be conducted by trained personnel.


Attach the Hanger Bar to the Overhead Lift HeliQ

Feed out enough of the belt and connect the Hanger Bar to the loop on the lift belt by threading the loop diagonally upwards until the tension of the belt is released, see figure below. The product is now ready for use.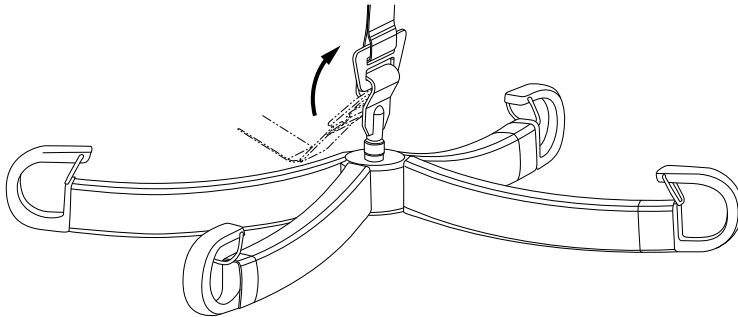


Detach the Hanger Bar from the Overhead Lift HeliQ

Feed out enough of the belt, until the tension of the belt is released, and disconnect the Hanger Bar from the loop on the lift belt by threading the loop diagonally downwards, see figure below. The product is now fully detached.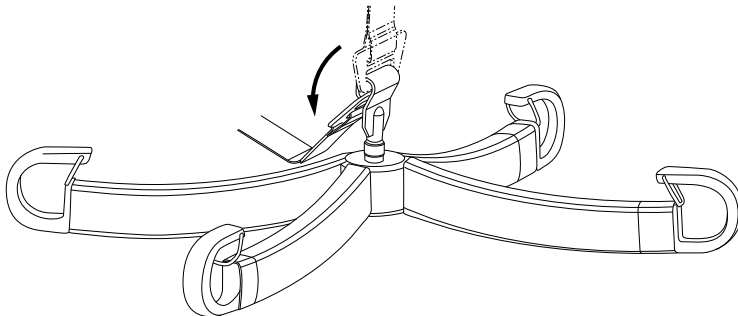


Sling attachment


Put the sling loop around the hook of the hanger bar (4x).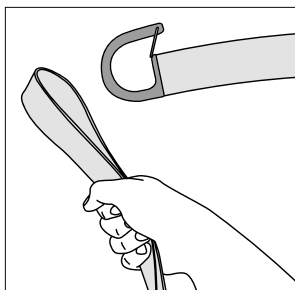


Bring every loop behind the safety latch at the hook.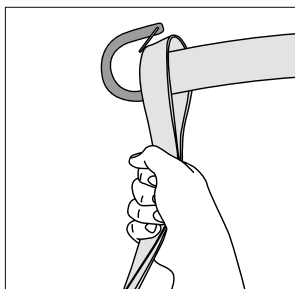

Check that all loops are safely attached to the 4 hooks.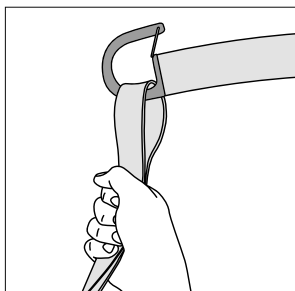

The person can now safely be lifted, according to the user instructions of the sling in use.

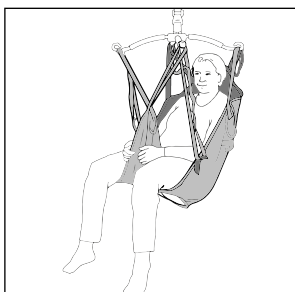Hanger Bar 4P HeliQ

Care instructions

Annual inspection

The Hanger Bar shall be inspected visually before every lift by the caregiver and at least once a year by someone with Human Care qualifications and in accordance with Human Care's directive. Pay particular attention to parts that show wear.

Cleaning

When cleaning the surface and hole of the Hanger Bar only use acid- and chloride-free detergents, like warm water or a soap solution, with a soft cotton swab. Carefully clean the hole. The cleaning sessions shall always end with water rinse and drying with a swab. The product shall be dry before attaching to the lift.

Disinfecting

If the Hanger Bar needs to be disinfected, the surface of the entire product can be cleaned using disinfectants, 70% ethanol, 45% isopropanol or similar should be used.

Service

Installation, service and maintenance shall only be performed by personnel with Human Care qualifications and in accordance with Human Care's directive. Only original spare parts may only be used.

Expected service life

The Hanger Bar is tested accordingly EN ISO 10535, which means it functions for 11 000 lifting cycles at maximum lifting capacity or for 10 years with correct service and maintenance.

Recycling

Once the product has reached the end of its technical life, it must be disposed of in constituent parts according to local authorities. Metal parts must be treated as metal waste for optimum recycling. Plastic parts can be sorted as residual waste.

The latest version of the instructions of use is always available at our website, www.humancaregroup.com.

Troubleshooting

Description	Action
The lifting hooks show a crack.	Replace the complete hanger bar for a new hanger bar. Discard the broken hanger bar.
Nothing happens when trying to lift a person.	Please go through the troubleshooting in specific lift user manual.
The loop slings cannot be detached from the Hanger Bar.	Make sure the locking clip is folded up when removing the loop slings.

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10
Email: info.se@humancaregroup.com

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone:- 613.723.6734
Fax:- 613.723.1058
Email: info.ca@humancaregroup.com

HUMAN CARE UNITED STATES

8006 Cameron Street, Suite K
Austin, TX 78754 USA
Phone: 512.476.7199
Fax: 512.476.7190
Email: info.us@humancaregroup.com

HUMAN CARE NEDERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577 412 171
Fax: +31 577 412 170
Email: info.nl@humancaregroup.com

www.humancaregroup.com