HUMAN CARE
Your Life. Your Way.

User manual / Handleiding

(Powered) 4-point spreader bar /
(Elektrisch) 4-punts kanteljuk

Part no: 92441 / 92442 / 92443

Contents

NL

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ENG

Safety instructions	13
Specifications	15
Product description	16
Intended use	17
Product label	17
Symbols on the product label and product	18
Functions	19
Control panels	19
Attach the 4-point Spreader Bar Small to the overhead lift	19
Attach the 4-point Spreader Bar Medium and Large to the overhead lift	20
Detach the 4-point Spreader Bar Small to the Overhead Lift	21
Detach the 4-point Spreader Bar Medium and Large to the Overhead Lift	21
Care instructions for the 4-point Spreader Bar	22
Troubleshooting	22

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ENG

Part no: 92441, 92442, 92443

The “WARNING!” triangle is used when particular attention is required.

Read these instructions carefully before using the product. It is important that the device is only used with the appropriate Human Care products.

Before use, always check that the device is not damaged or worn.

Warning: Product modification is not permitted.

Human Care’s products are continuously being developed and updated to provide our customers with the best products we reserve the right to make design changes without prior notice.

Safety instructions

ENG

Before using for the first time, always ensure that:

- You have read and understood the user manual for the lift and all of its accessories.
- The lift has been installed in accordance with the installation instructions.
- The lift accessories are intended for use with the lift and must be correctly fitted.
- The lift’s batteries have been charged for at least eight hours.
- Personnel, who use the equipment, have been given adequate training in the use of the (Powered) 4-point Spreader Bar and its accessories.


Important! The 4-point Spreader Bar is designed exclusively for lifting people, and only one person at a time.

WARNING: To avoid the risk of electric shock, this equipment must only be connected to a supply main with protective earth.

During use, always ensure that:

- The client is not left unattended during a lifting situation.
- The lift belt is not twisted or worn and that it moves freely in and out of the lift.
- Accessories, e.g. slings are showing no signs of wear or other damages.
- The lift is correctly suspended in the lift belt or distance strap.
- The right types, sizes, materials and design of the accessories have been selected by a professional with respect to the client's safety and need.
- The accessories have been correctly attached. The sling's clips are attached correctly to the mounting pins, this should be checked before lifting.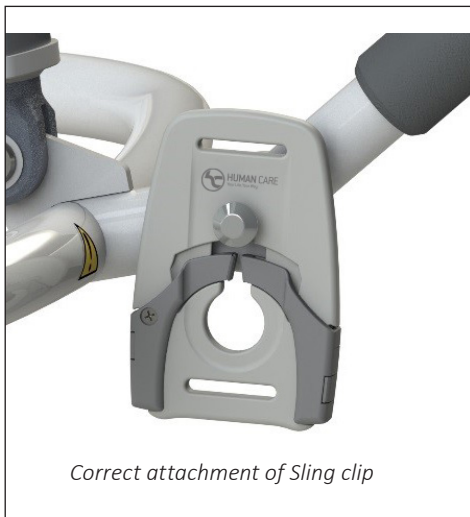

Important! Never attach the slings in any other way than displayed in the image, the textile loops are not for attaching to the Spreader Bar.

Human Care provides suitable training in safe operation of the (Powered) 4-point Spreader Bar and other accessories, further information about slings and other accessories can be found at www.humancaregroup.com.

Important! If the maximum load differs between lift, (Powered) 4-point Spreader Bar and slings, then the lowest maximum load shall always be used.


Important! To ensure that the lift belt will not be damaged, it is important that the (Powered) 4-point Spreader Bar is balanced and that the sling is suspended correctly. Never hold or pull the lift belt and (Powered) 4-point Spreader Bar when moving a patient. It is also important that the lift belt is tensioned as it is wound in.

Specifications

ENG


	Dimensions: (DxWxH)	Weight	Safe Working Load:
92441 4-point spreader bar Small	465x450x495 mm 18.3x17.7x19.4"	7,7 kg / 17,0 lbs	227 kg / 500 lbs
92442 Powered 4-point spreader bar Medium	465x590x495 mm 18.3x23.2x19.4"	10,6 kg / 23,4 lbs	227 kg / 500 lbs
92443 Powered 4-point spreader bar Large	515x730x495 mm 20.2x28.7x19.4"	10,9 kg / 24,0 lbs	227 kg / 500 lbs
Materials used	Steel		
Available accessories	Full line of Human Care Slings with clip attachment		
No. of possible connection points	4-points		
Input - powered version (M&L)	24V DC, 4 A		
Duty cycle (Medium & Large versions only)	10%, Max 2 min ON / 18 min OFF		
IP rating	IPx4		
CE approved	Yes, according to directive MDD93/42/EEC		
In compliance with	ISO 10535		
Noise level	<50 dB (A) loaded <42,4dB (A) unloaded		

Manual Version(Small)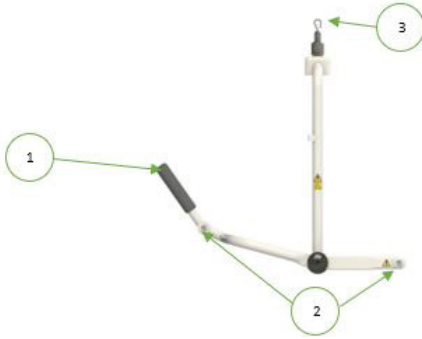

1	Handle
2	Sling suspension points
3	Attachment hook

Powered versions (Medium and Large)

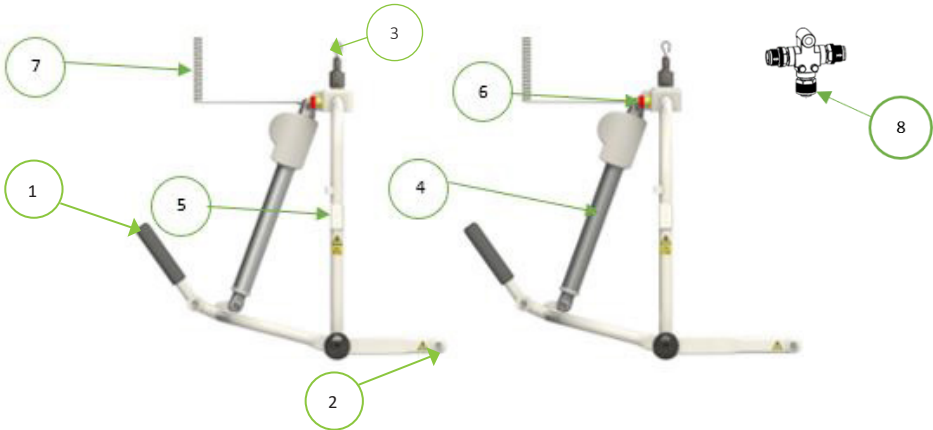1	Handle
2	Sling suspension points
3	Attachment hook
4	Linear Actuator
5	Control Panel for tilting
6	Emergency button
7	Power Supply Cable
8	T-splitter for power supply to HeliQ

Intended use

ENG

The 4-point spreader bar makes it possible to lift and reposition a client in the most comfortable and safe way for both the client and the caregiver, even during the actual lifting process.

The spreader bar shall only be used together with Human Care's overhead lift HeliQ.

The tilting movement is powered, which means that the needed efforts of the caregiver are reduced to a minimum, meaning lesser risks for work related injuries.

Intended use of the sizes:

- Small: for smaller persons and/or children, manual version.
- Medium: for adults and appropriate for most of the lifting situations, powered version.

- Large: for patients who require more space in the sling because of broad-shouldered, obese or sensitivity to pressure, powered version.

The Powered 4-point Spreader Bar has been designed and tested for use indoors and is classified as an IPX4 product.

The climatic conditions should be as follows: ambient temperature from 10°C to 40 °C, relative ambient humidity from 30% to 75% and ambient pressure from 700 hPa to 1060 hPa.

Important! The product must not be immersed in water.

Product label

ENG

The product label contains information about the (Powered) 4-point Spreader Bar and is located on the backside of the product.

The product has been tested by an accredited test institute and complies with the necessary requirements in accordance with the Medical Devices Directive for Class 1 products (MDD93/42/EEC).

The product is CE marked and complies with the requirements of EN 10535, EN 60601-1 (Electrical Safety) and EN 60601-1-2 (EMC).

4-point spreader bar  **HUMAN CARE**
Your Life Your Way.

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
UPC     Label 92362

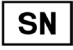www.humancaregroup.com


	<p>CE marking</p>
	<p>Date of manufacturing</p>
	<p>Manufacturer</p>
	<p>Max. load</p>
	<p>Instructions: Read the instructions and make sure you understand them before using this product</p>
	<p>Part number/reference number</p>
	<p>Serial number</p>
	<p>Warning</p>
	<p>The device contains electronics and it must be recycled appropriately</p>
	<p>Applied part type B: approved for physical contact with patients (IEC 60601-1)</p>

Hand maneuvering (A) (Model S, M & L)

The product is equipped with a handle to ease movement of the patient during use and enabling ergonomic rotation of the Spreader Bar.

Emergency stop (B) (Model M & L)

In case of an emergency, the emergency stop can be activated by pressing the RED button. To reset the emergency stop, turn the RED button clockwise.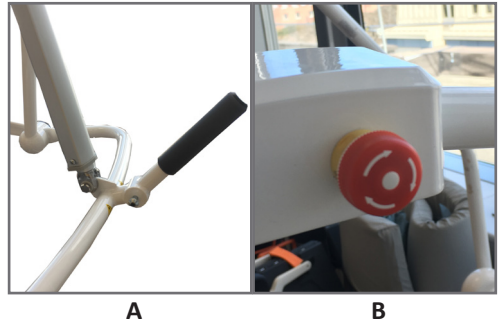


A

B


Control panels

The control panels, located on each side of the product (Medium and Large versions only), are equipped with two tilting buttons, one for moving the patient from laying to seating position (1) and one for moving the patient from seating to laying position (2). Each button need to be pressed continuously to achieve any movements.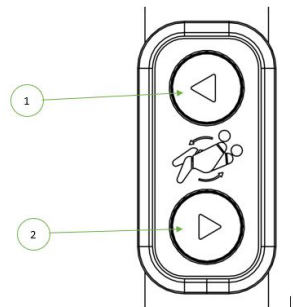


Attach the 4-point Spreader Bar Small to the overhead lift

Place the Spreader Bar on a suitable surface. Feed out enough of the lift belt and connect the Spreader Bar to the hook on the lift belt by threading the hook diagonally from below, see figure. The product is now ready for use.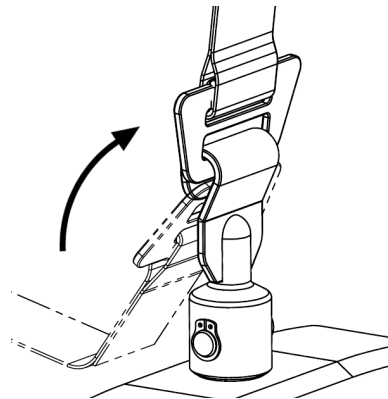


A: Power connection

Place the overhead lift (1) on a suitable surface. Connect the power supply cable (2) to the T-Splitter (3). If a power traverse lift is used connect it to the open end of the T-Splitter (4). Connect the T-Splitter to the Power Traverse contact on the overhead lift (5), see figure.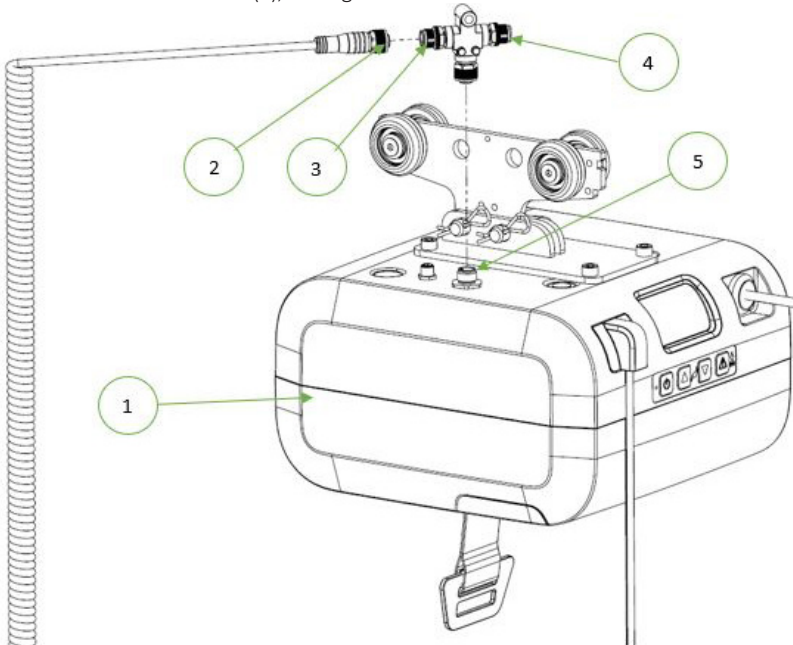

B: Attachement of the bar to hook of overhead lift


Place the Powered 4-point Spreader Bar on a suitable surface. Feed out enough of the lift belt and connect the Spreader Bar to the hook on the lift belt by threading the hook diagonally from below, see figure. Connect the power supply cable to the Spreader Bar. Switch on the main power on the overhead lift and release the Powered 4-point Spreader Bar emergency switch. The product is now ready for use.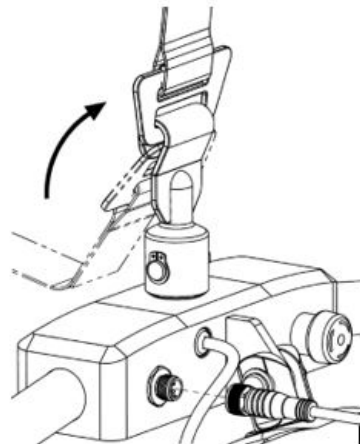

Detach the 4-point Spreader Bar Small to the Overhead Lift

ENG


Place the Spreader Bar on a suitable surface. Feed out enough of the lift belt and disconnect the Spreader Bar to the hook on the lift belt by threading the hook diagonally from above, see figure.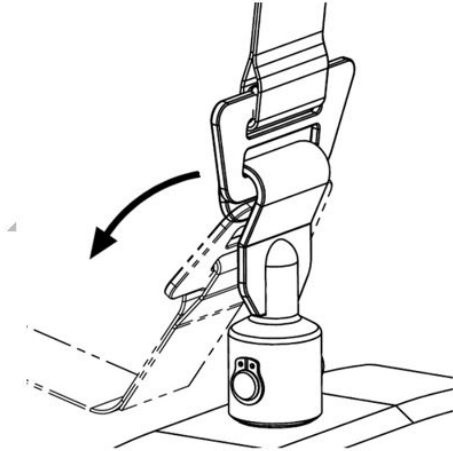

Detach the 4-point Spreader Bar Medium and Large to the Overhead Lift

ENG

Switch off the main power on the overhead lift. Disconnect the power supply cable of the Spreader Bar. Place the Spreader Bar on a suitable surface. Feed out enough of the belt and disconnect the Spreader Bar from the hook on the lift belt by threading the hook diagonally from above, see figure.

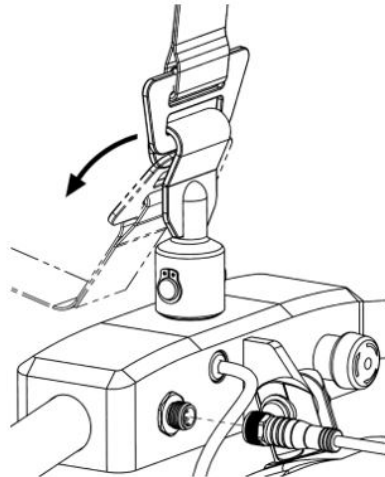Annual inspection

The 4-point Spreader Bar shall be inspected visually at least once a year by someone with Human Care qualifications and in accordance with Human Care's directive. Pay particular attention to parts that show wear.

Cleaning

When cleaning the surface of the 4-point Spreader Bar only use acid- and chloride-free detergents, like warm water or a soap solution, with a soft cotton swab. The cleaning sessions shall always end with water rinse and drying with a swab. The product shall be dry before attaching to the lift.

Disinfecting

If the 4-point Spreader Bar needs to be disinfected, the surface of the entire product can be cleaned using disinfectants, 70% ethanol, 45% isopropanol or similar should be used.

Service

Installation, service and maintenance shall only be performed by personnel with Human Care qualifications and in accordance with Human Care's directive. Only original spare parts may be used.

Expected service life

The 4-point Spreader Bar is tested according to EN ISO 10535, which means it functions for 11 000 lifting cycles at maximum lifting capacity or for 10 years with correct service and maintenance.

Recycling

Once the product has reached the end of its technical life, it must be disposed of in constituent parts according to local authorities. Metal parts must be treated as metal waste for optimum recycling. Plastic parts can be sorted as residual waste.

Troubleshooting

Description	Action
Nothing happens when you press the Up button or the Down button on the control panels.	<ol style="list-style-type: none"> 1) Check that the mains switch on the lift is on. 2) The lift batteries are fully discharged. Charge the lift immediately. 3) The emergency stop is not pushed in. Deactivate by turning the red button clockwise.
Nothing happens when trying to lift a person.	Please go through the troubleshooting in specific lift user manual.
The loop clips cannot be detached from the 4-point Spreader Bar.	Make sure to open the locking mechanism by pressing the buttons on the side of the clip.

For more information about our accessories and technical documentation you are welcome to visit our website, www.humancaregroup.com

The latest version of the user manual is always available at our website, www.humancaregroup.com.

HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B
117 43 Stockholm
Phone:- +46 8 665 35 00
Fax:- +46 8 665 35 10
E-post: info.se@humancaregroup.com
www.humancare.se

HUMAN CARE CANADA

10-155 Colonnade Road
Ottawa, ON K2E 7K1
Phone:- +1 613.723.6734
Fax:- +1 613.723.1058
Email: info.ca@humancaregroup.com
www.humancaregroup.com

HUMAN CARE UNITED STATES

Human Care USA Inc.
8006 Cameron Street, Suite K
Austin, TX 78754 USA
Phone: +1 512 476 7199
Fax: +1 512 476 7190
info.us@humancaregroup.com

HUMAN CARE AUSTRALIA

Unit 4, 49 Bertie Street
Port Melbourne, Victoria 3207
Phone:- +61 1300 132 736
Fax:- +61 (03) 9645 4954
Email: info.au@humancaregroup.com
www.humancaregroup.com

HUMAN CARE NETHERLAND

Elspeterweg 124
8076 PA Vierhouten
Phone: +31 577-412171
Fax: +31 577-412170
Email: info.nl@humancaregroup.com
www.humancaregroup.com

HUMAN CARE
Your Life. Your Way.