PM-5050

PM-5024/PM-5025

PM-5020 ohne Abbildung/without picture

PM-5030

Flexibles Rutschbrett Flexible Sliding Board

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of transfer aids.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The flexible Sliding Board consists of a machined, highly elastic, flexible polyethylene plate, which depending on the version consists of three parts and can be folded lengthwise. Anti-slip strips are attached to the bottom of the flexible Sliding Board to prevent the board from sliding away on the underlay. The flexible Sliding Board is available in 3 sizes and is used to transfer patients. The patient is no longer required to come to a standing position, but can complete the position change independently or with assistance while seated.

The flexible Sliding Boards are part of a product line which is used specifically to aid in the transfer of patients in hospitals and other health care facilities, as well as in home care.

The flexible Sliding Board is a safe transfer aid, which allows for a transfer from the bed to the wheelchair which is pleasant and comfortable for the patient as well as being simple and back-friendly for the user.

The product may only be used for the specified purpose and under the specified conditions.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item No.: PM-5020 flexible Children´s Sliding Board

Item No.: PM-5030 flexible Sliding Board with rounded edges

Item No.: PM-5050 flexible Sliding Board

Item No.: PM-5024 flexible Sliding Board

Item No.: PM-5025 flexible Sliding Board

1.2 Accessories

No accessories are available for this device.

1.3 Scope of delivery

Flexible Sliding Board

User manual

1.4 Storage, transport

Store/transport at -5 C – 40 C

Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

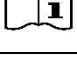
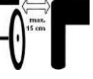
2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (200 kg).
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The handle is intended solely for positioning. Do **not** reach into the handle openings during transfer.
- The use of non-standard cleaning agents or bleaching agents, alcohols or other chemical additives may lead to permanent deterioration of the sliding mat properties.
- Ensure that both ends of the flexible Sliding Board are securely supported on the transfer surfaces. The clear distance bridged with the flexible Sliding Board must not exceed 20 cm (10 cm in the case of PM-5020).
- Transfers under naked skin must be aided with a protective layer (e.g. towel).
- The smooth surface significantly reduces resistance due to friction during repositioning. Check the transfer surfaces carefully to ensure they are fixed in place securely (e.g. wheelchair, bed).

2.1 Symbols used

	Wipe off		
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label
	The distance between free transfer surfaces using the Sliding Board must not be more than 20 cm (10 cm in the case of PM-5020/PM-5024/PM-5025, deviations see item 6).		max. load capacity see item 6

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

	Visual inspection Ensure that material, seams, handle loops and buckles are undamaged and in functional order.		Stability test Bend product to test stability
	Sliding properties Place the product on a firm surface Pull your hands over the product with pressure Good glide - easy gliding Low glide: no gliding/moving possible		
	➤ Defective products may no longer be used!		

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since flexible Sliding Boards are suitable for all applications involving people who require assistance.

- Seated transfer with more supporting surface area and reduced friction.
- Maintaining independence with restricted mobility.
- Facilitates position changes with and without assistance.
- Reduction of shear and friction forces in case of an elevated risk of decubitus.
- Protection from injuries due to bed edges, wheelchair wheels or brake levers.
- Good adaptability thanks to the flexible material when height differences must be overcome.

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Observe the safety information under item 2.

- All diagnoses which preclude sitting.

3.3 Application risks

- Insufficient care while placing under buttocks → risk of pinching the skin
- The movement is performed surprisingly or too quickly → risk of falling, particularly in the case of anxious patients, attention must be paid to the patient and his condition
- Lack of stabilisation → risk of falling

3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

3.5 Application illustration

3.6 Application

The flexible Sliding Board is used for transferring patients. The patient is no longer required to come to a standing position, but can complete the position change independently or with assistance while seated.

Before using, please note

- PM-5030: the back edge must be bent up before the transfer.
- PM-5050: the back edge must be bent up before the transfer, the front edge must also be bent down.
- Ensure that both ends of the flexible Sliding Board are securely supported on the transfer surfaces. The clear distance bridged with the flexible Sliding Board must not exceed 20 cm (for the PM-5020 = 10 cm).

3.6.1 Placement

- Steady the upper body and tilt slightly to the side.
- Push the flexible Sliding Board under the buttocks approx. one hand width, parallel with the sitting surface.
- Raise the patient.
- Turn the flexible Sliding Board underneath the patient in the desired direction to the transfer target.
- The transfer can take place.

3.6.2 Transfer methods

- Independently through weight transfer, do **not** reach into the handle openings as the handle is intended solely for positioning.

- Transfer with assistance.
- Transfer with additional aids.
- Backwards transfer in case of double amputation is possible.
- Transfer with additional sliding aids (BM-0652/-0752): this combination facilitates the transfer to and positioning in a wheelchair and may only be performed with assistance!

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products

Flexible Sliding Boards are used in combination with the following products:

- BM-0652/0752 Blue Motion® Sheets
- PM-3005 Leg Belt
- PM-6013/14/15 Support Belt
- PM-3012/15 Raising Aid

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

3.7.1 Sets

Item No.	Ingredients set
PM-5031	PM-5030 flexible Sliding Board
	BM-0652 Sheet

Please observe the user manuals of the other items as well.

4 Product marking / CE

The Petermann flexible Sliding Board is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

Cleaning	Disinfection (if necessary)
Damp cleaning	Recommended disinfectants for wipe disinfection:
Warm water / mild detergent	
Do not use abrasive or bleaching agents!	
	<ul style="list-style-type: none"> - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - Bacillo® 30 Foam - Bacillo® 30 Tissues - Bacillo® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues*

Do not use sharp-edged objects!

* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-5020	Polyethylene, latex free	black	200 kg with 10 cm free transfer surface
			170 kg with 20 cm free transfer surface
PM-5030	Polyethylene, latex free	black	200 kg
PM-5050	Polyethylene, latex free	black	200 kg
PM-5024	Polyethylene, latex free	black	200 kg
PM-5025	Polyethylene, latex free	black	200 kg

Item no.	Description	Dimensions in mm	Weight	UDI
PM-5020	Flexible Childrens's Sliding Board	580 x 320 x 6	1040 g	(01)04250355350206(10)x
PM-5030	Flexible Sliding Board with rounded edges	740 x 320 x 6	1280 g	(01)04250355350305(10)x
PM-5050	Flexible Sliding Board	740 x 320 x 6	1320 g	(01)04250355350503(10)x
PM-5024	Flexible Sliding Board	450 x 200 x 6	500 g	(01)04250355350244(10)x
PM-5025	Flexible Sliding Board	600 x 250 x 6	850 g	(01)04250355350251(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

8 Disposal information

There are no specific disposal regulations for flexible Sliding Boards, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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