User manual Crossover 90222

Rollator



Please read this user manual before assembling and using this rollator.

Instruction for use of the rollator

Parts Lists:

The rollator includes:

- 1) Height Adjustment Handle 2) Brake Lever 3) Red Pull Strap (Seat)
- 4) Red Height Adjustment Button 5) Wheels 6) Soft Bag 7) Backstrap



This rollator offers a sturdy seat for the user.

The rollator can be folded, making it easier for carrying or storage.

1. Folding and Unfolding the Rollator

To fold the rollator, grip the red pull strap that is attached to the seat, and pull towards you and upwards.

To unfold / open the rollator, push the handles apart and press down firmly on the seat platform until it is flat and locked.



2. Using the Brakes



The rollator is equipped with both a **walking brake** and a **parking brake**.

Walking Brake

To slow down the pace or to come to a stop, use the walking brake.

Squeeze the brake levers up to the handles to stop. The wheels will move freely once you release the brake levers.

Parking Brake (Engage/Disengage)

To secure the rollator from rolling away or to sit down on the seat, push the brake lever down until they lock and with an audible "click" sound. Pull up on the brake lever to unlock the parking brake.

While you are sitting on the rollator, always ensure the parking brake is engaged.




3. Handle Height Adjustment

To adjust the handles, press the red button firmly, to disengage the lock. The handle height can be adjusted in 1" increments. While pressing the red button, pull the handle upwards or press downwards to the next height adjustment. As you are pulling up or pushing down the handle, release the red button, and it will lock, with an audible "click", when it reaches the next height increment. Repeat this step until you reach your optimal hand position when using the rollator. Ensure both handles are set to the same height.

4. Backstrap Installation

Insert the backstrap end onto rollator handle mount. To remove, slide the backstrap up and out of the handle mount.

By tightening the screws, the backstrap can be fastened permanently. Now the backstrap is fixed and cannot be easily removed.

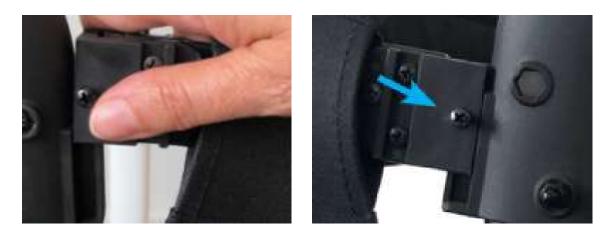

5. Soft Bag Installation



Hang the straps of the bag over the mounted rods on the side of the seat.

Accessories:



Foam Cushioned Pads/ Coussinets en mousse Part. no 90232



Cane/Umbrella Holder/ Support pour canne ou parapluie Part no. 90230



Slow-Down Brake/ Frein de ralentissement Part. no 90231

Specifications

	Crossover / 90222		
1	Overall Length	73	cm
2	Overall Width	66	cm
3	Handle Height min	86	cm
4	Handle Height Max	98.5	cm
5	Seat height	53cm/21in	cm/in
6	Seat size	45x20	cm
7	Fold width	24	cm
8	Width between handles	48	cm
9	Front wheel	20cm/8in	cm/in
10	Rear wheel	20cm/8in	cm/in
11	Net weight	8	kgs
12	Carton size	74 x 25 x 88	cm
13	Max user weight	136 kgs/300lbs	kgs/lbs
14	Bag loading	5	kgs

Care and Maintenance:

- 1. Occasionally wipe the rollator with a soft cloth, do not use harsh abrasive chemicals.
- 2. Do not store the rollator in damp environments.
- 3. During assembly of the rollator, pay attention to possible pinch points to avoid injury.
- 4. While adjusting the handle height, do not grip the adjusting tube.
- 5. Regularly check to ensure brake operation is normal.
- 6. Periodically check that all the screws, nuts, and bolts are securely tightened.
- 7. Do not hang anything on the handles or backstrap.
- 8. Make sure the brakes are engaged and locked while seated.
- 9. Use only authorized parts for repair of this product.
- 10. Never use this rollator for any other reason than its intended purpose.

Distributed by :



www.humancaregroup.com

Manufacturer:

Ralon Medical Equipment CO., Ltd.

Add: XiLian Development Zone, Danzao, Nanhai, Foshan, Guangdong, China.

Distributed by:

Human Care Canada

Phone: - 613.723.6734

Email: info.ca@humancaregroup.com

99999, Rev 02, User manual Crossover, Oct 2021