

# Full Body Sling E-clip



Comfortable fit

Optimal support

Safe E-clips



## Suitable for many lifting situations

Our Full Body Sling E-clip is a comfortable sling that is ideal for people who like full body support during the lifting process. This sling is available with integrated headrest or with removable stiffeners in case this sling is used for longer periods of time (in-situ).



Thanks to its special shape, the Full Body Sling E-clip provides a comfortable support to the head, torso and thighs, keeping the client in a safe posture. This sling will position the client into a reclining position, reducing the pressure on their legs. This can also be a pleasant posture for clients with reduced core stability and/or with a tendency to overstretch. In a reclined position, the client will not be able slip from the sling. Thanks to the integrated, reinforced headrest, this sling can even be used to lift up clients from flat surfaces, such as the floor. The slings are designed for four-point suspension from a spreader bar and are available in 2 materials.

**Polyester net** is a **soft**, thin material that breathes and dries quickly. It is mildly elastic and shapes itself to the client's body. The sling can stay behind the client for a short period of time. An additional advantage is that the sling is very suitable for showering and bathing.

**Soft Air Mesh** is a lightweight, cool, soft and breathable fabric with moisture-wicking properties, which means it draws moisture away from the skin through the fabric where it can evaporate quickly. Another advantage of this material is that it is anti-bacterial for improved hygiene. The sling has "two-way stretch" properties, making it feel almost as a second skin.



Mobility Aids



Healthcare Beds



Lifting Solutions



Hygiene



Convertible Chairs

## Accessories



### Spreader bar

For HeliQ stationary lift, available in Small (manual), Medium and Large (electric). Art. No. 92441, 92442 and 92443.




### Spreader bar

For Diana II Comfort 4ESB (electric). Art. No. 90560.



### Spreader bar

For Diana II/ Diana II Comfort 4MSB (manual), Art. no. 90558/ 90559.

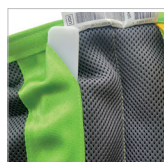

### In-Situ

The Full Body Sling In-Situ E-clip has removable ribs in case the sling remains behind the patient. Art. no. 25011C

## Use




### From and into bed

Suitable for lifting clients from bed and helping them sit down on bed.



### From and to a chair

Suitable for positioning in a chair and for transfers in and out of a chair.

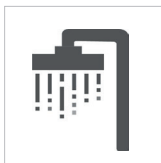

### Showering


The Polyester Mesh material is highly suitable for wet environments.



### To and from the floor

Designed for lifts from the floor.

## Features

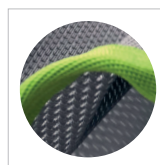


### Polyester Net

A soft and thin material that breathes and dries quickly. Ideal for showering and bathing. Article. No 25005C.




### E-clip integrated lock

Can be used one-handed. Universally compatible with all lifts equipped with mushroom fasteners Ø 6.8-8.2 mm.

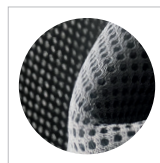

### Soft Air Mesh



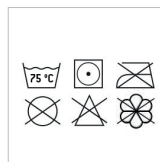





Great pressure-relief. Excellent pressure distribution thanks to the 3D-structure of the fabric. Prevents and protects. Art. No. 25010C/25011C




### Cleaning and care instructions

For cleaning and care instructions, see the label of the sling. Wash in an approved laundry bag.

## Technical specifications

Product name	Article number	Size	Color	A	B	C
Full Body Sling E-clip Net JS	25005C-JS	Junior Small		16 cm / 6.3"	63 cm / 24.8"	76 cm / 29.9"
Full Body Sling E-clip Soft JS	25010C-JS					
Full Body Sling In-Situ E-clip JS	25011C-JS					
Full Body Sling In-Situ E-clip Net JS	25006C-JS					
Full Body Sling E-clip Net JM	25005C-JM	Junior Medium		19 cm / 7.5"	67 cm / 26.4"	80 cm / 31.5"
Full Body Sling E-clip Soft JM	25010C-JM					
Full Body Sling In-Situ E-clip JM	25011C-JM					
Full Body Sling In-Situ E-clip Net JM	25006C-JM					
Full Body Sling E-clip Net S	25005C-S	Small		31 cm / 13.8"	85 cm / 30.3"	90 cm / 35.4"
Full Body Sling E-clip Soft S	25010C-S					
Full Body Sling In-Situ E-clip S	25011C-S					
Full Body Sling In-Situ E-clip Net S	25006C-S					
Full Body Sling E-clip Net M	25005C-M	Medium		40 cm / 15.7"	92 cm / 34.6"	91 cm / 35.8"
Full Body Sling E-clip Soft M	25010C-M					
Full Body Sling In-Situ E-clip M	25011C-M					
Full Body Sling In-Situ E-clip Net M	25006C-M					
Full Body Sling E-clip Net L	25005C-L	Large		40 cm / 17.3"	98 cm / 38.6"	100 cm / 39.3"
Full Body Sling E-clip Soft L	25010C-L					
Full Body Sling In-Situ E-clip L	25011C-L					
Full Body Sling In-Situ E-clip Net L	25006C-L					
Full Body Sling E-clip Net XL	25005C-XL	Extra Large		44 cm / 18.9"	105 cm / 41.3"	102 cm / 40.1"
Full Body Sling E-clip Soft JS	25010C-XL					
Full Body Sling In-Situ E-clip XL	25011C-XL					
Full Body Sling In-Situ E-clip Net XL	25006C-XL					
Full Body Sling E-clip Net XXL	25005C-XXL	Extra Large [2x]		44 cm / 18.9"	111 cm / 43.7"	106 cm / 41.7"
Full Body Sling E-clip Soft XXL	25010C-XXL					
Full Body Sling In-Situ E-clip XXL	25011C-XXL					
Full Body Sling In-Situ E-clip Net XXL	25006C-XXL					
Full Body Sling E-clip Net XXXL	25005C-XXXL	Extra Large [3x]		44 cm / 18.9"	118 cm / 46.4"	108 cm / 42.5"
Full Body Sling E-clip Soft XXXL	25010C-XXXL					
Full Body Sling In-Situ E-clip XXXL	25011C-XXXL					
Full Body Sling In-Situ E-clip Net XXXL	25006C-XXXL					
Maximum user weight		300 kg / 660 lbs				

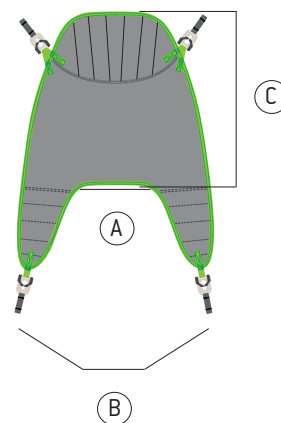

### HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B  
117 43 Stockholm  
+46 8 665 35 00  
info@humancaregroup.com

### HUMAN CARE CANADA

10-155 Colonnade Road  
ON K2E 7K1 Ottawa  
+1 613 723 6734  
info.ca@humancaregroup.com

### HUMAN CARE THE NETHERLANDS

Elspeterweg 124  
8076 PA Vierhouten  
+31 577 41 21 71  
info.nl@humancaregroup.com

### HUMAN CARE UNITED STATES

8006 Cameron rd, Suite K  
TX 78754 Austin  
+1 512 76 7199  
info.us@humancaregroup.com