HUMAN CARE
Your Life. Your Way.

User manual / Bruksanvisning /
Notice d'utilisation / Benutzerhandbuch /
Handleiding

Slings / Selar / Harnais / Gurte / Tilbanden
Limb Sling 25700, Limb Sling NSB-010

Content

ENGLISH

User Manual	3
Safety Instructions	4
General Description / Intended Use	5
Technical Specifications	6
Product Label	7
Compliances and Standards	8
Suspension Alternatives	8
Hangar Bars for Human Care Lifts	9
Compatibility	10
Sling Application	11
Troubleshooting	13
Care Instructions	14

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User Manual

Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

INFORMATION symbol indicates recommendations and information for proper, trouble-free usage.

WEIGHT CAPACITY symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

CE MARKING symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

MEDICAL DEVICE symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

RECYCLE according to local regulations.

READ INSTRUCTIONS and ensure you fully understand them before using this product.

Product modification is not permitted.

Before use, always check that the product is not damaged or worn.

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at www.humancaregroup.com.

Contact your local distributor if you have any questions about the product and its use.

Notice to user/patient in case of serious incident

Any serious incident that has occurred in relation to the product, should be reported to the local contact, who reports to the manufacturer, and the competent authority of the country in which the user/patient is established.

Slings


Safety Instructions


The process of lifting and moving a person always involves risks. Therefore, you must read the user manuals for the lift and the lifting equipment carefully. Always make sure that the lifting equipment you are using is intended for use with the lift. As a care provider, you are responsible for the client's safety and you must know whether the client will be able to cope with the lifting process. The sling must be fitted to meet the client's specific needs. If you are in any doubt, please contact the supplier.

- Carry out a careful visual inspection of the sling every time you use it.
- Check that the right sling is being used for the client. The model, size and material must meet the client's needs.

- Plan the moving and lifting process so that it is as safe as possible.
- Before raising the client, check that the sling is correctly attached to the lift/hanger bar once the lift straps are stretched.
- It is important that the hooks of the hanger bar are in the correct position, with the opening of the hook facing upwards on both sides to securely hold the straps in place. Recheck the position of the hooks once the lift straps are stretched.
- It is important to always check that the loops are at the bottom of all hooks (see pictures) and securely placed below the locking pin on all hooks.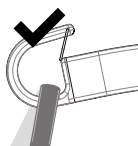

Correct

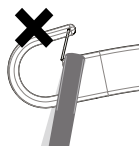Incorrect

- Check that the client is sitting securely in the sling before starting to lift him/her.
- Check that the lift is balanced when moving, raising and/or lowering the client. If you are using a Human Care lift with telescopic hanger bar, it is important that the bar hooks are in the same position on both sides.
- Our slings have been designed and tested for indoor use with one client at a time and must only be used for transferring a person. This product is not a swing.
- Make sure no sharp objects are in contact with the sling.

General Description / Intended Use

Limb Sling

Our Limb Sling is designed to comfortably and conveniently assist in lifting, supporting and positioning a client's body part like legs, arms, shoulders, pannus or hips. This allows better access for treatment, performing surgery, leg/venous wrapping, foot care, wound care treatment, hygiene and a variety of other care related tasks.

With a cooperative client, who has good trunk control, endurance and cognition, it can also help in turning, for elevating a shoulder for access to the head, hair, scalp etc., or for elevating hips for a slight pelvic bridge position. The limited amount of material allows easy application by a single caregiver for these types of applications.

The limb sling can be used for manual lifting, but it is preferable when used for lifting with an overhead lift or floor lift to promote caregiver safety and avoid patient injury and/or skin friction. It can also be used to support the pannus (manual lifting).

The Limb Sling can be used to support patient movement while standing up from a chair or

bed by placing around their shoulders as well as to lift a limb while getting out of bed.

Caregiver safety is an important concern in facilities. Using the Limb Sling reduces the caregiver's physical load by lessening their movements of bending and leaning forward as it acts as an extension for their arms while transferring the patient's body part to the correct location and height. It can be used under either one or both legs and can be used in conjunction with a slide board/sheet or a soft turntable while getting out of a bed. In some cases, due to limb size or technique, it may be advantageous to utilize two Limb Slings to provide maximum support.

The Limb Sling (25700) consists of:



- Limb Sling 25700

The Limb Sling (NSB-010) consists of:

- Limb Sling NSB-010

Slings

Technical Specifications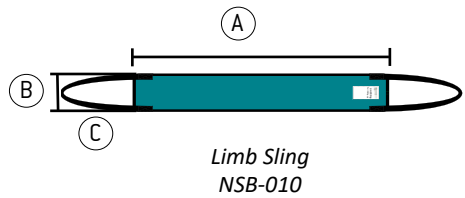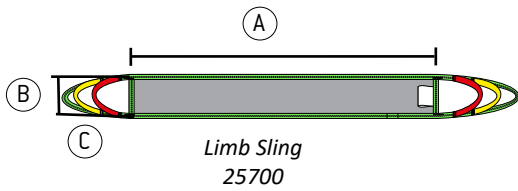

Limb Sling

Item	Part No.:	Size	Description	A (cm/inch)	B (cm/inch)	C (cm/inch)	Weight Capacity (kg/lbs)
Limb Sling	NSB-010	Medium	Non-colored loop	122/48	17.5/6.9	45.5/18	200/441
Limb Sling	25700-M	Medium	Colored loops	123/48.4	17/ 6.7	36/14	200/441

**The weights given are only guidelines and are not a substitute for individual tests. Discrepancies can occur, tolerances ± 1 cm, 0,3".*

Product Label

The label on the sling contains the following information:

1. Human Care company logo
2. This way up arrow and marking the centre line of the sling
3. CE and MD marking
4. Product name and size
5. "This side facing out" – outside of the sling, not in touch with the client
6. Part number
7. Safe Working Load
8. Manufacturer
9. Date and country of production
10. Instruction labels
 - a. Do not use fabric softener
 - b. Visual inspection
 - c. Latex Free
 - d. Read the manual before use
11. Serial number
12. Washing instructions
 - a. Machine Wash Temperature
 - b. Do Not Bleach
 - c. Tumble Dry, Low Heat
 - d. Do Not Dry Clean
 - e. Never Iron
13. Owner name. (For example, write the client's name or another unique identifier)
14. Indicate bi-annual inspection
15. UDI: (01) EAN/GS1/GTIN (11) manufacturing date (21) serial number
16. Unique Product Code

1 HUMAN CARE
Your Life. Your Way.

2 UP

3 CE
MD

4 **Limb Sling, Medium**

5 This way out, back

6 REF 25700-M

7 SWL < 200kg / < 441lbs

8 Human Care HC AB
Arstaängsvägen
21B 117 43 Stockholm
Sweden
<http://www.humancaregroup.com>

9 2023-01-10

10 a. Do not use fabric softener
b. Visual inspection
c. Latex Free
d. Read the manual before use

11 SN *335486010001*

12 a. Machine Wash Temperature
b. Do Not Bleach
c. Tumble Dry, Low Heat
d. Do Not Dry Clean
e. Never Iron

13 75°C / 167°F
#

14 Due date next periodic inspection (every 6 months), mark:

2023	1	2	3	4	5	6	7	8	9	10	11	12
2024	1	2	3	4	5	6	7	8	9	10	11	12
2025	1	2	3	4	5	6	7	8	9	10	11	12
2026	1	2	3	4	5	6	7	8	9	10	11	12

15 UDI (01) 0 7350011 05851 8 (11) 230110 (21) 335486010001

16 UPC 8 1608 90030 1

Slings

Compliances and Standards

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This sling has been tested in accordance with ISO 10535.

Suspension Alternatives

Our slings are designed with the premise that all people are different with varying lifting requirements, needs or desires.

To make all our clients feel as comfortable as possible in our slings, most of them can be used with 2 or 3-point suspensions. Some slings can also be used with a 4-point suspension. Below the options are displayed.

2-point suspension

Suitable for most lifting situations providing a good seating position. 2-point suspension is available with floor lifts, fixed and portable ceiling lifts.

Two-point suspension with crossed leg support:

Thread the strap on one leg section through the strap on the other leg section before attaching them to the hanger bar.

Two-point suspension with overlapping leg support:



This method is suitable for lifting clients with amputated legs, for example. Extension straps can be attached to the lifting straps to prevent the client from leaning too far back in the seated position.

Two-point suspension with separate leg support:

Put the client's legs in each leg section and attach the lifting straps to the hanger bar. Note: This method increases the risk of the client sliding out of the sling


An individual assessment of the client's needs must always be carried out before choosing an alternative connection method.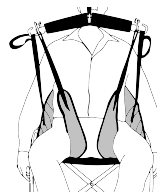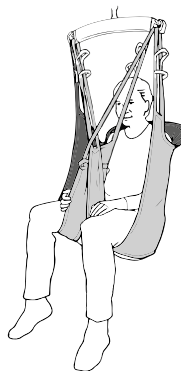

3-point suspension


Offers an elongated sitting position and more even weight distribution. The 3-point suspension is suitable especially for clients carrying an increased risk of pressure sores or who need additional space for the upper body.

Thread the left leg section through the loop on the right leg section. Attach the leg section lifting straps to the leg hooks and the back section lifting straps to the lift.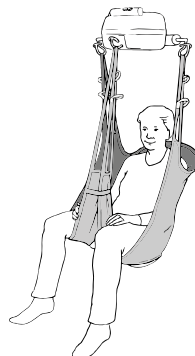

4-point suspension

A spacious sitting position making it suitable for larger clients or when using a stretcher. 4-point suspension is available with both fixed and portable ceiling lifts or floor lifts.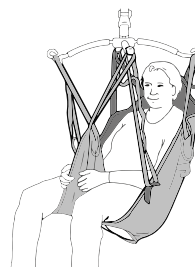

Hanger Bars for Human Care Lifts

HeliQ
(7275x)

Roomer S / Altair
(55751 45 cm, 17,7" / 55764 55 cm, 21,7")

HeliQ
(92938)

Accessory	Description	Part no.
2-point hanger bar	For stationary lift HeliQ. 3 widths:	35 cm/13,8": 72754 45 cm/17,7": 72755 60 cm/23,6": 72757
2-point hanger bar	For portable lifts Roomer S / Altair, 2 widths:	45 cm/17,7": 55751 55 cm/21,7": 55764
4-point hanger bar	For stationary lift HeliQ	92938
4-point suspension	4-point suspension accessory (49 cm). Enables 4-point lifting with the Altair and Roomer S lifts. The suspension bar is placed on the lift's single leg hook.	55619

Compatibility

Human Care Group slings are designed for use with all Human Care lifts and hanger bars. The products are also compatible with products of other suppliers utilizing the same attachment method for securing slings to a hanger bar and lift. However, due to the wide range of products worldwide, Human Care cannot be responsible for any errors that may result from improper use or application of combinations of slings, hanger bars and lifts of other suppliers. The use of the combined system is at the prescriber's own risk and liability.

To ensure the safety of the client and caregiver, Human Care recommends the following minimum requirements:

1. When using slings with lifts and hanger bars, made by manufacturers other than Human Care, the combined system needs to be designed with the same sling attachment method for the hanger bar of floor lifts and ceiling lifts, meaning loop style slings are combined with a loop style hanger bar, clip style slings are combined with a clip style hanger bar.
2. The involved patient lifts and sling must be CE marked in accordance with the medical device directive, MDD 93/42/EEG and/or MDR 2017/75 and comply

with standard requirements for patient lifts as stated in EN ISO 10535.

3. The caregiver must follow the manufacturer's instruction manuals and recommendations with respect to the use, care, client, caregiver, cleaning, and inspection of the slings and lifts.
4. Different products on the assembled lift system: i.e., lift, hanger bar, sling, scale and other accessories may have different maximum loads. It is always the lowest maximum allowable load indicated for the respective products and the system that applies for the entire system. Always check the markings on all individual products in the system.
5. Individual risk assessment, including a practical test of the system and validation with the specific combination used are required to affirm that the size of the sling is correct for the intended use and compatible to the width and design of the hanger bar. The combined system needs to be documented by the competent assessor.

If there are any questions, please contact your local Human Care office or distributor.

Sling Application

Limb Sling


1. Apply Limb Sling by sliding it under the patient or by using the standard roll technique. It is recommended that the sling be folded in half or one-third intervals with the loop folded under the sling.

2. The sling should be inserted under the limb in areas of limited leg-bed contact, for example the back of the knee. Make sure the sling loop is not touching the skin.

3. Once the sling is in place the folded portion of the leg strap can be removed gently, avoiding skin contact. Pull it out on the opposite side and connect the loops to the hanger bar.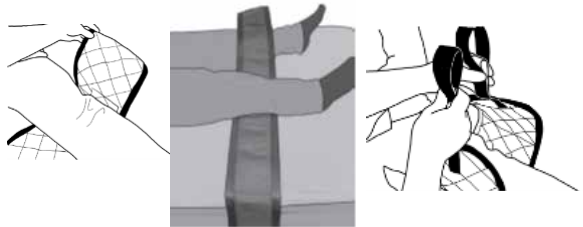

4. Raise the lift slightly and then pause to ensure that both the sling loops are secure in the hanger bar and that the patient is not experiencing any pain or discomfort. If everything is okay, continue lifting to the desired position.

Troubleshooting

Description	Action
The client is at risk of sliding out	<ul style="list-style-type: none"> The sling may be too large. The sling is not correctly fitted- the length of the loops should be stretched at the same time.
The client is unbalanced when lifted	<ul style="list-style-type: none"> The instructions on the individual test log have not been followed or a new assessment needs to be carried out.
The sling is too small	<ul style="list-style-type: none"> This can be uncomfortable for the client. The sling can cut into the client's flesh and/or his/her back may not have enough support.

Note: It is important that the right sling is used for each client and that it is individually tested to ensure that it meets the client's need and the transfer situation.

Care Instructions

Check for wear and tear

Check the sling for wear and tear. It is important to check the fabric, the edging, the seams and the lifting straps.

Damaged lifting equipment must not be used.

Cleaning

Human Care slings are intended for individual use. To avoid infections or transmission of bacteria, it is important to clean the slings properly. All slings should be machine washed and dried as soon as possible. Slings should always be cleaned after use in a wet situation.

Always refer to the label of the sling before washing.

Periodic inspection

A periodic inspection must be carried out at least every 6 months. See Human Care's periodic inspection log which can be found on our website at www.humancaregroup.com

Product lifetime

The product's expected lifetime 3 years or 150-180 times washing, if the product is used as intended and maintained according to the manufacturer's instructions, depending on the intensity of use and maximum load applied during use. If the product label is no longer legible, the product should be discarded.

Warranty and support

If you need information or support, please contact your Human Care office, www.humancaregroup.com, or distributor.

Recycling

Once the product has reached the end of its life, it should be disposed according to your country's rules, especially for the separate collection of electrical and electronic products.

Correct disposal helps prevent negative consequences for the environment and human health.

Overview of Sling Material Care		Polyester	Polyester Net	Trevira / Soft Air Mesh	Polyester/Polyester Net	Polyester/Cotton	Ventilated Polyester	Cotton	Polyester Tricot	Coated Nylon
	Machine Wash Maximum Temperature 60°C (140°F) (slings with padding)	●	●							●
	Machine Wash Maximum Temperature 75°C (167°F)	●	●	●	●		●		●	
	Tumble Dry Low, Maximum Temperature 30°C- 54°C (86°F- 130°F)									●
	Tumble Dry Low, Maximum Temperature 60°C (140°F)	●	●	●	●		●		●	
	Do Not Tumble Dry					●		●		
	Do Not Iron	●	●	●	●	●	●	●	●	●
	Do Not Dry Clean	●	●	●	●	●	●	●	●	●
	Do Not Bleach	●	●	●	●	●	●	●	●	●
	Do Not Use Fabric Softener	●	●	●	●	●	●	●	●	●
	Do Not Wash					●		●		
	Dispose of After Use					●		●		
	Disinfection Permitted (see list of approved disinfectants)	●	●		●		●		●	

Disinfection Procedure

Human Care slings are intended to be used with one patient at a time. To avoid possible transmission of infectious diseases, it is important to clean the sling if it has become soiled or if it is to be used by a different patient.

Approved Disinfectants	Solutions to Avoid
Chemical Disinfection: 70% Ethanol, 45% isopropanol or similar	 Steam sterilization
Thermal Disinfection: Wash 65°C (149°F) for a minimum of 10 minutes ¹	 Iodine based disinfectants
Thermal Disinfection: Wash 71°C (160°F) for a minimum of 3 minutes ¹	 Gas sterilization
Note: Add 8 minutes to both thermal disinfection options to ensure heat has penetrated the fabric. Finish with a rinse cycle. ¹	

¹ Ref: HSG (95) 18: Hospital Laundry Arrangements for Used and Infected Linen, Department of Health, NHS (National Health Service, England)

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